
HEINE HL1200® Untersuchungsleuchte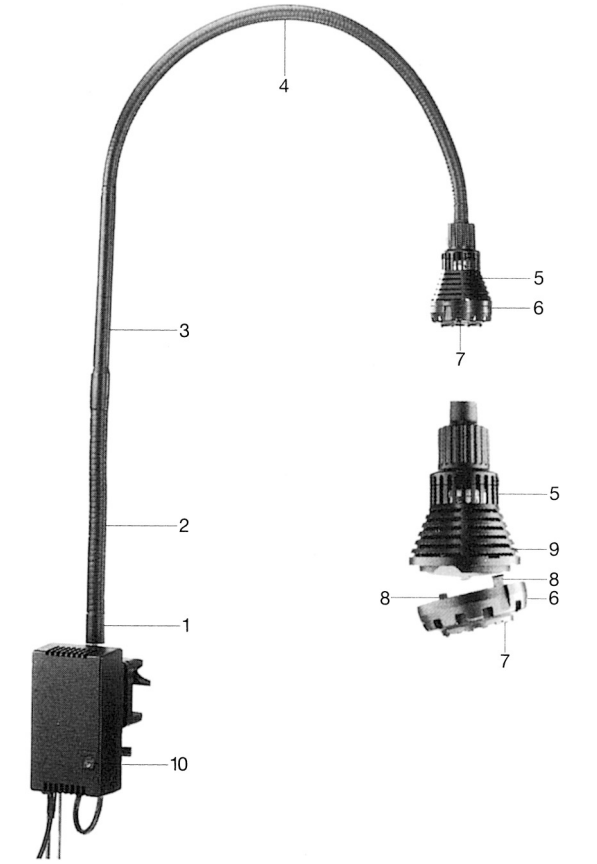

CE 93/42/EWG/CEE/EEC

HEINE

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49(0)8152/38-0 · Fax +49(0)8152/38-202
E-Mail: info@heine.com · www.heine.com
med 5395 1/11.08

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HEINE HL1200® Untersuchungsleuchte

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Instructions

HEINE HL1200® Examination Light

Before using the HEINE HL1200® read these instructions carefully and keep them in a safe place for future reference.

In accordance with local regulations this product should be disposed of as an electronic device separately .

Safety information

Caution: Do not use in the presence of fluids. When servicing use only identical replacement parts

Danger: Explosion hazard. Do not use in the presence of flammable anaesthetics. Indoor use only.

Regulations for use: The HL1200 halogen examination light is suitable for general illumination in a medical environment in clinic and examination rooms and can be left switched on for extended periods.

Safety: Do not open the transformer housing. Apart from a bulb change, all repairs must be carried out by qualified personnel. The HL1200 should only be used when completely assembled.

Assembly: The standard version of the HL 1200 with direct connection to the transformer housing (1) can be mounted in any position. The version with a rotating mount (1) must be mounted vertically. Ensure that the clamp is firmly attached. Do not obstruct the cooling slits in the transformer housing.

Safety in use: The mains voltage shown on the nameplate must correspond to the actual mains voltage. The HL1200 may only be used with the protective cap (6) fitted to the illumination head. The illumination head must not be covered by foil or cloth etc. The light should not be used in environments where an explosion hazard is present. Do not shine the light directly into the eyes. Position the illumination head at least 20 cm from the object.

Assembly/Installation

Installation instructions and materials are included in the packaging.

- Wall-mounting (instructions S-249.07.01)
- Clamp-fitting (instructions S-249.23.01)
- Wheeled stand (instructions S-249.33.01)

Plug in the mains cord only after assembly is completed.

Instructions for use

The swivel arm of the HL1200 has two flexible (2, 4) and one rigid section (3). The flexible sections can be bent in any direction at up to 90° (2) or 180° (4). They should not be bent to a greater degree to avoid wear. To position the light, grasp the flexible section behind the illumination head (5).

Switch on by means of switch (10).

Bulb change

- Disconnect the mains cord plug.
- Allow the bulb to cool.
- With thumb and forefinger press the cap together at the two marks (7) and ease off the cap (6) outwards and down.
- With thumb and forefinger grasp the two points on the bulb and pull it out of the socket.
- When inserting the new bulb, ensure that the contact pins are not bent. Do not touch the safety cap.
- To replace the cap, engage the clip (8) in the opening marked (●) on the illumination head and press the cap (6) until the second clip (8) clicks into place.

Important note! The safe and reliable operation of the light can only be guaranteed if an original HEINE bulb is fitted.

Changing a fuse

Disconnect the mains cord plug! The fuses are located in the fuse compartment of the integral plug. Use only the fuses listed in the technical specification.

Cleaning

Disconnect the mains cord plug! Clean only when the unit is cool. To clean the HL1200 use only a soft cloth with soapy water or other non-aggressive solution.

Technical specification

Input:	115 V~/60 Hz (①)	Fuse	2 x 400 mA slow
(see nameplate)	240 V~/50/60 Hz (②)		2 x 200 mA slow
Power consumption:	40 VA		
Output:	12 V / 20 W		
Bulb:	12 V / 20 W		
Light beam angle:	10°		
Bulb life:	approx. 4000 hours		
Colour Temperature:	approx. 3200 K		
Conditions of use:	continuous (ambient temperature up to + 40 °C)		
Classification:	Type B †		
Weight with wall mount:	2,4 kg		
Weight of clamp:	0,2 kg		
Weight of wheeled stand:	3,2 kg		
Length of Flexible arm:	114 cm		
International Protection:	IP 20		
Equipment:	Class I		

Accessories

Clamp (black)	J-000.27.010
Clamp (white)	J-000.27.012
Wheeled Stand (not shown)	Y-096.50.004
Reflector lamp 12V/20W	J-005.27.075

Applicable Standards: ① **UL 2601-1**
 ② **EN 60 601-1**

Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries).

We warrant the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered.

The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

Bought from	_____
date	_____

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