HEINE LED LoupeLight®

CE 93/42/EWG/CEE/EEC

HEINE

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. + 49 (0) 81 52 / 38 - 0 · Fax + 49 (0) 81 52 / 38 - 202
E-mail: info@heine.com · www.heine.com
med 0909 1 / 2.10

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DEUTSCH

HEINE LED LoupeLight®

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ENGLISH

HEINE LED LoupeLight®

Before using the HEINE LED LoupeLight®, carefully read through these Instructions for Use and keep them for later reference.

  In accordance with local regulations this product should be disposed of as an electronic device separately.

Safety Information

Range of applications: The HEINE LED LoupeLight® is only intended for illuminating the field of view when used with the HEINE HR®-, HRP®- and HR-C®-Binocular Loupes.

Safety in use: Only operate the LED LoupeLight with the HEINE mPack LL®. Avoid any strong mechanical loading or kinking of electric cables.

General Safety: Do not shine light directly into the eyes to avoid dazzle, especially if the pupils are dilated.

Never ever hold magnifying glasses or any other optical system in front of the light of the LED LoupeLight.

For applications of the HEINE LED LoupeLight® in dental medicine: A yellow filter can be used to avoid premature hardening of composite material.

How to fit the LED LoupeLight to the binocular loupe mount without HEINE i-View®

- Loosen the screw (1) in the clamp (2) so that it easily slips onto the middle of the optics mount (3). The LED LoupeLight housing (4) has to **fit between** the eyepieces (**not above them**). Re-tighten the screw (1).
- The cable for the LED LoupeLight can be fixed with the cable clip on the right or left side of the frame as desired.

How to fit the LED LoupeLight to the binocular loupe mount with HEINE i-View®

- Unscrew the screw (1) on the light mount (2) so that this can be attached to the round attachment of the loupe mount (3). The LED LoupeLight housing has to **fit between** the eyepieces (**not above them**).
- The cable for the LED LoupeLight can be fixed with the cable clip (8) on the right or left side of the frame as desired

Fastening the electric cable to clothing

- Using the support clip that comes with the unit, you can fasten the cable to your clothing.

Setting up

- Connect the LED LoupeLight to the mPack LL.
- Switch on the LED LoupeLight and adjust vertically the light housing such that the illumination is centered on the area to be examined.
- You can control the brightness by adjusting swivel lever.
- Adjust the binocular loupe for the intended application. Be sure the eyepieces are positioned as symmetrically as possible on the optics holder (take off the loupe and check this), as otherwise the spot of light will appear shifted to the side.
- **LED LoupeLight on loupe mount without HEINE i-View®.** If required, a steri-lever can be attached to the two pins (6) so as to swivel the binocular loupe along with the light (9).
- **LED LoupeLight on loupe mount with HEINE i-View®.** If required, a steri-lever can be attached to one end (7) of the loupe mount so that the binocular loupe can be swivelled independently of the light (only HR 2,5/HRP Loupes).

Be sure to note:

The direction and angle of the illumination can only be adjusted vertically, not horizontally.

Care Instructions

The lens systems of the HEINE LED LoupeLight® are water- and dust-resistant (comply with IP 65/IEC 60529 [2000] specification). The light housing is not water-restant and should be cleaned with a micro-fibre. For the housing and cable, use a cloth moistened with alcohol or a non-aggressive agent. Steri-levers can be sterilised as desired.

Technical Specifications

Length of the cable:	LED LoupeLight 1,80 m
Diameter of the light spot at a distance of 420 mm:	approx. 80 mm
Vertical direction of illumination:	± 7 ° adjustable
Classification:	moderate risk, IEC 62471 [2006]

Spare Parts and Accessories

Sterilisable Swivel Levers (with HEINE i-View®)	C-000.32.551
Sterilisable Swivel Levers (without HEINE i-View®)	C-000.32.524
Yellow filter	C-000.38.241

Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries). We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded. For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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