BM 19

- Ⓧ **Gebrauchsanleitung**
Sprechendes Oberarm-Blutdruckmessgerät (2–10)
- Ⓜ **Instruction for Use**
Speaking upper arm blood pressure monitor (11–18)
- Ⓧ **Mode d'emploi**
Tensiomètre parlant pour bras (19–27)
- Ⓧ **Instrucciones para el uso**
Tensiómetro para brazo con voz (28–36)
- Ⓧ **Instruzioni per l'uso**
Sfigmomanometro parlante da braccio (37–45)
- Ⓧ **Kullanma Talimatı**
Konuşan üst koldantansiyon ölçüm cihazı (46–53)
- Ⓧ **Инструкция по применению**
Говорящий прибор для измерения
артериального давления в плечевой артерии.. (54–63)
- Ⓧ **Instrukcja obsługi**
Mówiący ciśnieniomierz na ramię (64–72)

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Dear Customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for the applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air.

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Best regards,
Your Beurer Team

1. Getting to know your instrument

The upper arm blood pressure monitor is used for non-invasive measurement and monitoring of adults' arterial blood pressure. You can use it to measure your blood pressure quickly and easily, storing the results and displaying the progression of readings together with the average.

A warning is issued for anyone suffering from cardiac arrhythmia. The values determined are classified and graphically evaluated according to WHO guidelines.

The device also has an acoustic voice output.

Keep these instructions carefully for further use and also let other users have access to them.

2. Important information

Advice on use

- Always measure your blood pressure at the same time of day, in order to ensure that values are comparable.

- Relax for approx. 5 minutes before each measurement.
- You should wait five minutes before measuring a second time.
- The readings that you take may only be used for information purposes – they are not a substitute for a medical examination! Discuss your results with the doctor. Never use them to make medical decisions independently (e.g. regarding medication and dosage thereof).
- There may be incorrect measurements where there is cardiocirculatory disease, as well as with very low blood pressure, circulatory disorders, dysrhythmia and other preexisting diseases.
- Only use the instrument on persons whose upper arm has the right circumference for the instrument.
- You can operate the blood pressure monitor with both batteries and mains. Note that it is only possible to save data when your blood pressure monitor is receiving power. As soon as the batteries wear out or you disconnect the mains adapter from the mains, the blood-pressure instrument loses the date and time.
- The automatic switch-off function switches off the blood pressure monitor in order to preserve the batteries, if no button is pressed within one minute.

Storage and Care

- The blood pressure monitor is made up of precision electronic components. Accuracy of readings and the instrument's service life depend on careful handling.
 - You should protect the device from impact, moisture, dirt, major temperature fluctuations and direct exposure to the sun's rays.

- Never drop the device.
- Do not use near strong electromagnetic fields, i.e. keep it away from any radio systems and mobile phones.
- Only ever use the cuffs provided with the monitor or original replacement cuffs. Otherwise erroneous results will be recorded.
- Do not press any buttons until the cuff is in position.
- If the instrument is not used for any length of time, we recommend removing the batteries.

Advice on batteries


- Batteries can be fatal if swallowed. You should therefore store the batteries and products where they are inaccessible to small children. If a battery has been swallowed, call a doctor immediately.
- Batteries should not be charged or reactivated with any other means, nor should they be taken apart, thrown in the fire or short-circuited.
- Remove the batteries from the instrument if they are worn out or if you are not going to use the instrument for any length of time. This prevents any damage as a result of leakage. Always replace all the batteries at the same time.
- Never use different types of battery, battery brands or batteries with different capacities. You should preferably use alkaline batteries.

Repair and disposal

- Batteries do not belong in domestic refuse. Used batteries should be disposed of at the collection points provided.
- Never open the instrument. If these instructions are not heeded, the warranty will be null and void.

- Never attempt to repair the instrument or adjust it yourself. We can no longer guarantee perfect functioning if you do.
- Repairs may only be performed by Beurer Customer Service or authorized dealers. However, always check the batteries and replace them if necessary prior to making any complaint.
- The appliance should be disposed of according to Regulation 2002/96/EC-WEEE (Waste Electrical and Electronic Equipment). In case of queries, please contact the municipal authorities responsible for waste disposal in your area. 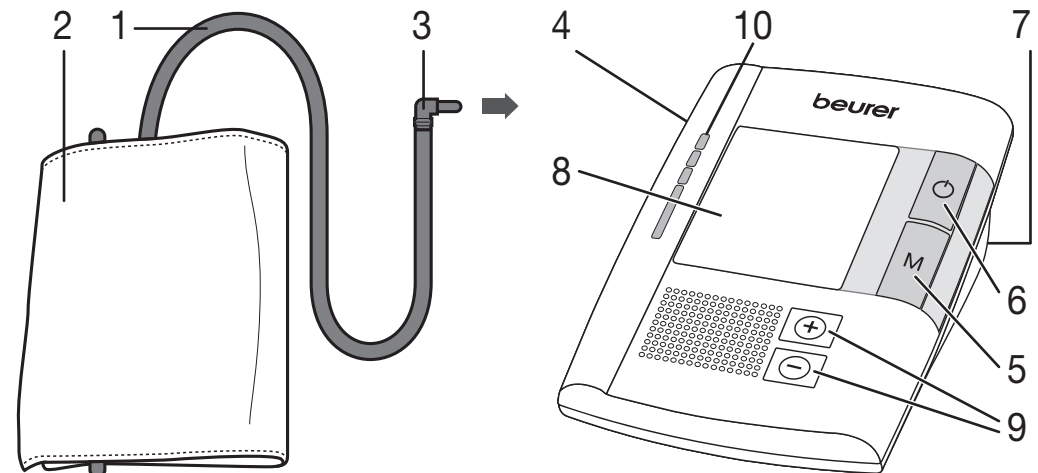

3. Unit description


1. Cuff tube
2. Cuff
3. Cuff connector
4. Cuff connector port (left side)
5. Memory button M
6. ON/OFF button 
7. Power socket (right side)

8. Display
9. Volume +/-
10. WHO-LED

Icons in the display:

1. "Change battery" icon 
2. Error icon 
3. Systolic pressure
4. Cardiac arrhythmia icon 
5. mmHg unit
6. Icon for users 1, 2
7. Diastolic pressure
8. Time and date
9. Number of memory space
10. Pulse icon 
11. Measured pulse
12. WHO - classification

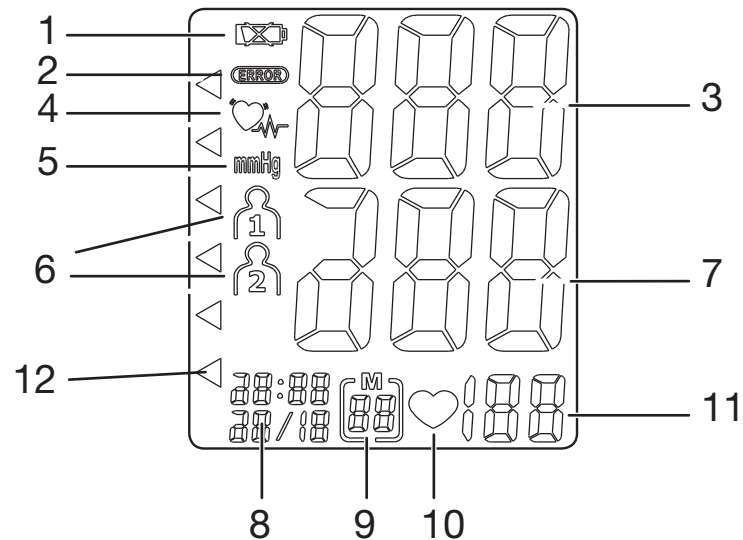
4. Prepare measurement

Inserting battery

- Remove the battery cover from the back of the monitor.
- Insert four AA 1.5 V alkaline batteries, making absolutely sure that you insert them with the correct polarity as marked. Never use rechargeable batteries.
- Replace the battery cover carefully.


4xAA(LR6) 1,5 V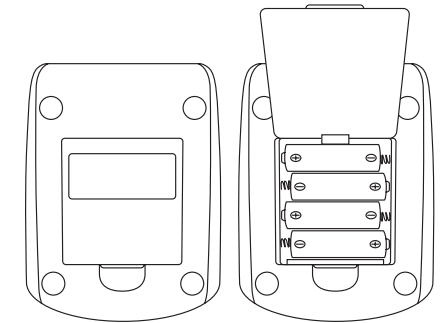

If the battery change  is continuously illuminated, measurement is no longer possible and you must replace all the batteries. Once batteries have been removed from the device, the time must be reset. The stored measured values are last.

Used batteries do not belong in the household waste. You are legally obliged to dispose of the batteries. Dispose of them via your specialist electrical supplier or local collecting point for recyclable waste.

Note: Batteries containing pollutant substances are marked as follows: Pb = Battery contains lead, Cd = Battery contains cadmium, Hg = Battery contains mercury.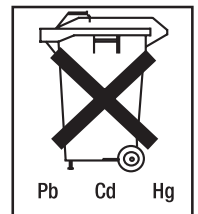

Setting date, time and language

It is vital to set the date and time. This is the only way that measurements can be saved and recalled with the correct date and time.

The time is shown in 24 hour format. The device also has five languages. The device is set to German on delivery. Proceed as follows to set the date, time and language:

- Press the memory key (M) for over 5 seconds.
- The day starts flashing. Use the +/- key to set the day from 1 to 31 and confirm with the memory key (M).
- Set the month, hour and minute and confirm in each case with the memory key (M).
- After setting the date and time you can set the language.
- Use the + and - volume keys to select the following languages: L1 = German, L2 = English, L3 = French, L4 = Spanish, L5 = Russian, L0 = no language. Confirm your selection with the memory key (M).
- You can then set the volume of the selected language: Vol 3 = high, Vol 2 = medium, Vol 1 = low. Confirm your selection with the memory key (M).

Note: Language off (L0) means that voice output is not selected and the measurement value will not be announced.


Mains operation

You can also operate this device with a mains adapter. No batteries should be in the battery compartment for this. The mains adapter is available from retailers or from the service address under order no. 071.13. The blood pressure monitor should only be used with the mains adapters specified here. The mains adapter should only be connected to the mains voltage indicated on the rating plate. Once you have disconnected the device from the mains, the date and time on the blood pressure monitor are lost. Any measurement results that have been stored are however retained.


5. Measuring blood pressure

Positioning cuff


Fit the cuff round your bare left upper arm. Blood circulation in the arm should not be restricted by tight clothing or other objects.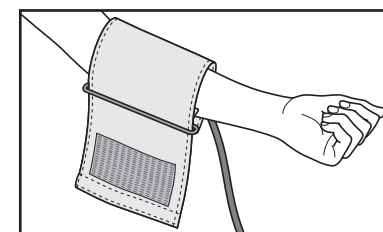


The cuff should be placed on the upper arm so that the lower edge is 2 to 3 cm above the bend of the elbow and above the artery. The tube should be in line with the centre of the palm.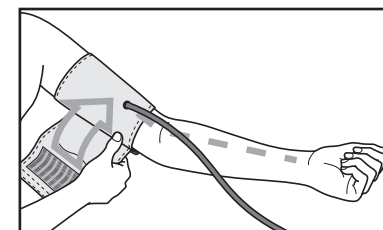

Now place the free end of the cuff snugly, but not too tightly, around the arm, and fix it with the Velcro fastener. The cuff should be fitted tight enough to allow just two fingers to fit beneath the cuff.

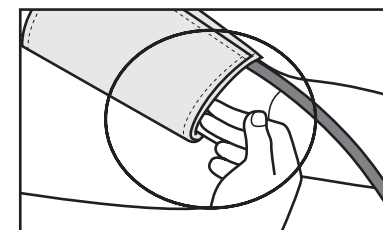
Insert the cuff tubing into the socket for the cuff attachment.

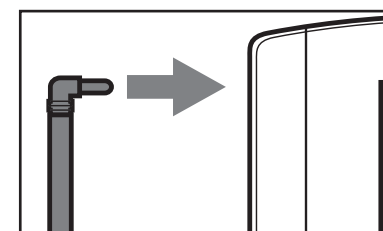Important: The instrument should only be operated with the original cuff. The cuff is suitable for an arm circumference of 23 to 33 cm.

Correct posture

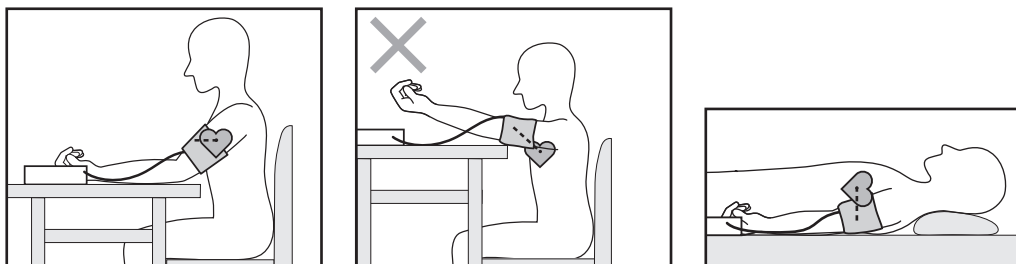- Rest for approx. 5 minutes before each measurement. Otherwise there may be divergences.
- You can perform the measurement either sitting or lying down. Always make sure that the cuff is on a level with your heart.
- In order not to distort the result, it is important to keep still during the measurement and not talk.

Select memory

Switch on the device with the  button. When the device is switched off only the clock in the display is shown. After the announcement you can select the desired memory space by pressing the + or - volume button.

Two memories with 60 memory spaces each are available to save the measurement results of two different people separately or to save morning and evening measurement values separately.

Measuring blood pressure

- Put on the cuff as described previously and assume the position in which you want to carry out the measurement.
- Start the measurement process by pressing ON/OFF button . After checking the display with all digits lighting up, the monitor will automatically inflate. Following the self-test, during which all display elements briefly appear, the measure-

ment begins. During inflation the device already determines measured values used to estimate the required inflation pressure. If this pressure is insufficient, the device automatically inflates by another 40 mmHg (Real Fuzzy Logic).

- Then the pressure in the cuff is slowly released and the pulse is measured.
- When the measurement has been completed, the remaining air pressure is released very quickly. The pulse, the systolic and the diastolic blood pressure are displayed.
- You can adjust the volume during the announcement by pressing + and -.
- You can interrupt measurement at any time by pressing the ON/OFF button .
- The **ERROR** icon appears if it has not been possible to perform the measurement properly. Observe the section in these instructions on error messages/troubleshooting and repeat the measurement.
- The device switches off automatically after 1 minute.

Wait at least 5 minutes before taking another measurement!

Evaluating results

Cardiac arrhythmia:

This instrument can identify possible cardiac arrhythmia disorders during measurement and if necessary indicates the measurement with the flashing icon .

This may be an indicator for arrhythmia. Arrhythmia is a condition where the heart rhythm is abnormal as a result of defects in the bioelectrical system controlling the heart beat. The symptoms (omitted or premature heart beats, slow or excessively fast heart rate) may be caused, among other things, by heart disease, age, physical predisposition, excessive use

of stimulants, stress or lack of sleep. Arrhythmia can only be ascertained through examination by your doctor.

Repeat the measurement if the flashing icon  is displayed after the measurement. Please note that you should rest for 5 minutes between measurements and not talk or move during the measurement. If the icon  appears often, please contact your doctor. Any self-diagnosis and treatment based

on the test results may be dangerous. It is vital to follow your doctor's instructions.

WHO classification:

According to WHO Guidelines/Definitions and the latest findings, the test results can be classified and evaluated according to the following chart:

Range of blood pressure values	Systolic (in mmHg)	Diastolic (in mmHg)	Measure	LED is lightning
Grade 3: Severe hypertension	>= 180	>= 110	Seek medical advice	red
Grade 2: Moderate hypertension	160–179	100–109	Seek medical advice	orange
Grade 1: Mild hypertension	140–159	90–99	Have it checked regularly by doctor	yellow
High-normal	130–139	85–89	Have it checked regularly by doctor	green
Normal	120–129	80–84	Check it yourself	green
Optimal	< 120	< 80	Check it yourself	green

Source: WHO, 1999

The bar graph in the display and the scale on the unit indicate the range of the blood pressure which has been recorded. If the values for systolic and diastolic pressure are in two different WHO ranges (e.g. systolic in the high-normal range and diastolic pressure in the normal range) the graphic WHO classification on the unit indicates the higher range (high-normal in the example described).

6. Saving, retrieving and deleting results

- The results of each successful measurement are stored together with date and time. With more than 60 items of measured data, the earliest items of data measured are lost.
- In order to retrieve measurement results again, press the memory key (M). First the average value of all saved measurements

of this user memory is displayed (Display "A"). ("). If you continue pressing the memory button 'M', what are in each case the last results measured are displayed and audibly announced with graphic WHO classification. You can adjust the volume during the announcement by pressing the volume key + and -.

- You can clear the memory by pressing and holding + and - simultaneously for 5 seconds after pressing the memory key (M) and selecting the user.
- If you wish to change the user memory, see "Select memory".

7. Cleaning and storing the instrument

- Clean your device and cuff carefully only with a slightly moistened cloth.
- Do not use detergents or solvents.

- On no account must you immerse the computer in water, otherwise liquid can enter it and cause damage.
- When storing the device, make sure that no heavy objects are placed on top of it. Remove the batteries. The cuff tube should not have any sharp kinks.

8. Error messages/troubleshooting

In case of faults, the **ERROR** message appears in the display. Error messages can occur when

1. inflation pressure is higher than 300 mmHg,
2. blood pressure is unusually high or low,
3. you move or talk during the measurement (in addition to **ERROR**, the cardiac arrhythmia icon  appears in the display),
4. the cuff tube is not inserted properly,
5. the cuff is inflated for longer than 16 seconds.

In the above cases, you must repeat the measurement. Make sure that the cuff tube is properly inserted and that you do not move or talk. Re-insert the batteries if necessary, or else replace them.

9. Specifications

Model No.	BM 19
Method of Measurement	Oscillometric, non-invasive blood pressure measurement on the upper arm
Range of Measurement	Pressure 0–300 mmHg, systolic 40–280 mmHg, diastolic 40–280 mmHg, pulse 40–199 beats/minute

Accuracy of the pressure reading	systolic ± 3 mmHg/ diastolic ± 3 mmHg/ pulse ± 5 % of the displayed value
Measuring uncertainty	max. permissible standard deviation according to clinical testing: systolic: 8 mmHg/diastolic: 8 mmHg
Sets of Memory	2 x 60 memory places
Unit Dimension	L147 x W104 x H66 mm
Unit Weight	Approx. 310 g
Cuff Size	cuff for upper arm circumference of 23–33 cm
Operating Temperature	+10 °C to +40 °C, ≤ 85 % RH
Storage-and transporttemperature	-20 °C to +70 °C, ≤ 85 % RH
Power Supply	4 x 1,5 V AA batteries (alkaline type LR6)
Battery Life	For approx. 250 measurements, depending on blood pressure level or inflation pressure
Accessories	With storage pouch, instruction manual, 4 pcs “AA” battery
Classification	Application part Type BF
Protection class	internal power supply, IPX0, no AP or APG, continuous operation
Key to symbols	Important! Read the operating instruction! Application part Type BF  

These specifications are subject to change without notice for purpose of improvement.

Adapter

Model No.	FW7333SM06
Input	100-240V, 50-60Hz
Output	6V DC, 1150mA, only in connection with beurer blood pressure monitor
Supplier	Friwo Gerätebau GmbH
Protection	This device is double insulated and protected against short circuit and overload by a primary thermal fuse. The green LED lights under normal conditions. Make sure to take the batteries out of the compartment before using the adapter
	double insulated / equipment class 2
Enclosures and Protective Covers	Equipment enclosed to protect against contact with live parts, and with parts which can become live (finger, pin, hook test). The operator shall not contact the patient and the output plug of AC adaptor simultaneously.

- This unit is in line with European Standard EN60601-1-2 and is subject to particular precautions with regard to electromagnetic compatibility (EMC). Please note that portable and mobile HF communication systems may interfere with this in-

strument. For more details, please contact customer service at the address indicated.

- The instrument is in line with the EU Medical Products Directive 93/42/EC, the German medical products act and European standards EN1060-1 (non-invasive blood pressure monitors, Part 1: general requirements) and EN1060-3 (non-invasive blood pressure monitors, Part 3: Supplementary Requirements for Electromechanical Blood Pressure Measurement Systems).
- If you are using the instrument for commercial or economic purposes, you must check measurement precision regularly in accordance with the 'Operators' Ordinance on Medical Products'. Even in the case of private use, we recommend checking measurement precision every two years at the manufacturers.

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