CLA9 - CLA9T

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CLA9 (T)

CONTENTS

CONTENTS	1
INTRODUCTION	2
Unpacking the CLA9/CLA9T	3
DESCRIPTION	4
General Description	4
INSTALLATION	6
Setting Up	6
USING THE CLA9/CLA9T	7
Turning the CLA9/CLA9T On and Off	7
Tone and Balance Setting	7
TROUBLESHOOTING	8
SAFETY INFORMATION	9
GUARANTEE	10
RECYCLING DIRECTIVES	12

INTRODUCTION

Congratulations on purchasing your Geemarc CLA9/CLA9T. The CLA9/CLA9T is a rechargeable assistive listening device. It amplifies speech together with sounds from a variety of sources like TV, radio and telephone.

The CLA9/CLA9T can be used as an alternative to Hearing Aids when using the headphones. If you use an inductive neck loop instead of the headset, the built in T-coil in the CLA9T enables the device to act as a complement to your Hearing Aid.

Note: users of digital hearing aids must make sure their aids have been programmed for use with the 'T' setting.

Its is important that you read the instructions below in order to use your Geemarc CLA9 or CLA9T to their full potential. Keep this user guide in a safe place for future reference.

INTRODUCTION

Unpacking the CLA9 and CLA9T



When unpacking the CLA9/CLA9T, you should find the following in the box:

- CLA9 or CLA9T Conversation Piece
- Headset with rubber ear cushions – 18mm diameter
- A pair of rubber ear cushions – 12mm diameter
- 2.4V (4/5) AAA Rechargeable Battery
- Travel charger
- Black velvet carrying case for either the headset and CLA9/CLA9T Conversation Piece
- A microphone with "holder" and 3m lead (**CLA9T only**)
- A lanyard
- A belt clip
- 1 User Guide


DESCRIPTION

General Description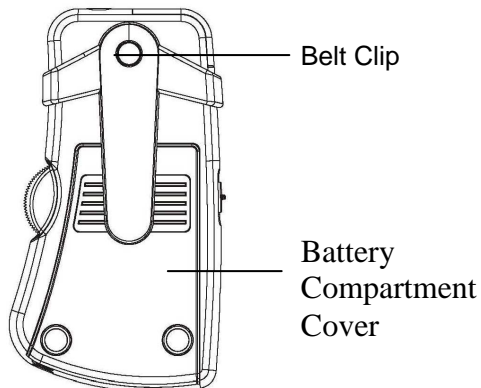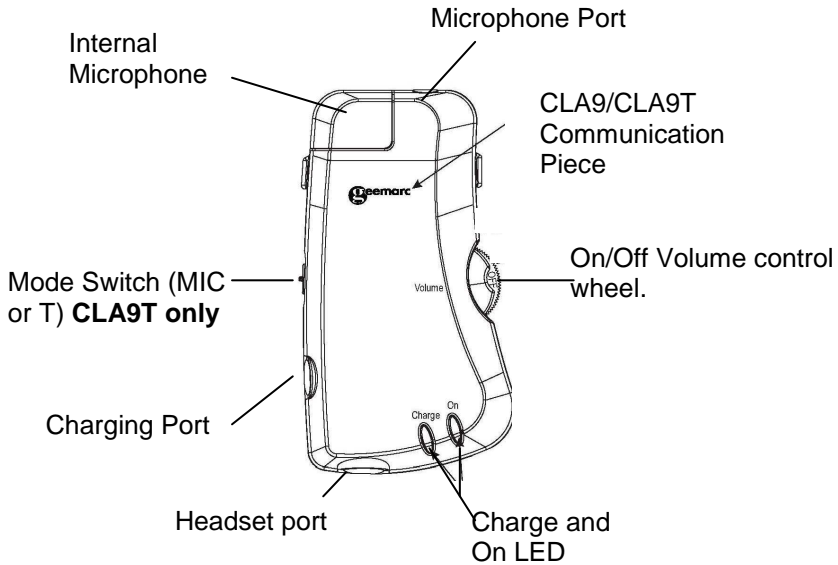

DESCRIPTION

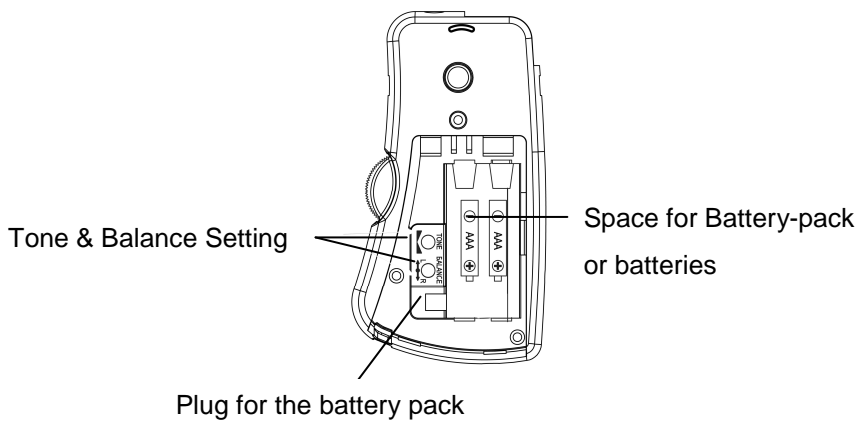INSTALLATION

Setting Up

Open the battery compartment. Insert the supplied rechargeable battery (2.4V) into the battery compartment located on the rear of the CLA9/CLA9T Communication Piece.

You have also the possibility to insert 2 singles batteries (1.5V AAA non rechargeable) instead of the one supplied .

Plug the battery charger into the mains supply and plug the other end of the lead into the charging port on the left hand side of the CLA9/CLA9T Communication Piece. Switch the mains supply on.

If the 'On' LED turns red when in use, the CLA9/CLA9T needs recharging, or replace new batteries if standard AAA are installed . During charging the LED turns on green , when fully charged , the LED is off.

Headset

Insert the headset lead into the headset port found on the base of the Communication Piece.

USING THE CLA9/CLA9T

Turning the CLA9/CLA9T On and Off

Put the headset on. Ensure the mode switch is set to MIC (CLA9T). Roll the On/Off Volume control wheel upwards. The On LED will display Green. Adjust the Volume to a comfortable level by rolling the On/Off Volume control wheel.

To turn the CLA9/CLA9T off or on , roll the On/Off Volume control wheel downwards. The Green LED will switch off and OFF will be showing on the On/Off Volume control wheel

CLA9T only

The CLA9T can be used in rooms especially equipped with a magnetic loop (conference ...etc) due to its "built in T coil". Simply by setting the mode switch to T, the CLA9T will receive the emitted magnetic waves then amplify them to your headset. You will receive all the benefits of the "built in T coil".

An external microphone is supplied and can be connected in the socket "Mic" (option for the CLA9), then set the switch to "Mic" position , in this case the "built in T coil" is disabled

Microphone

The CLA9/CLA9T are built with an internal microphone which enables you to hear all the surrounding sounds.

Tone and Balance Setting

The tone and Balance can be adjusted by using the controls found within the battery compartment.

TROUBLESHOOTING

The 'On' LED is Red

- Charge the battery if the battery pack is used or replace with new batteries if standard AAA batteries are installed.

The CLA9/CLA9T appears not to be working

- Make sure that the headset is securely plugged in
- Make sure that the volume is correctly adjusted
- Make sure that the battery is fully charged or use good batteries
- Make sure the mode switch is set to MIC if using headset/earpiece or T if using an induction loop (CLA9T)

The CLA9/CLA9T Is not charging

- Make sure that the power supply cord is plugged in correctly i.e. into a working wall socket and into the CLA9/CLA9T Communication Piece
- Make sure that the battery is inserted correctly and is of the recommended type

SAFETY INFORMATION

General

Do not open the unit. Contact the helpline for all repairs.

Cleaning

Clean the CLA9/CLA9T with a soft cloth. Never use polishes or cleaning agents - they could damage the finish or the electrics inside.

Environmental

Do not expose to direct sunlight.

Always ensure there is a free flow of air over the surfaces of the CLA9/CLA9T.

Do not place any part of your product in water and do not use it in damp or humid conditions e.g. bathrooms.

Do not expose your product to fire or other hazardous conditions.

Avoid touching the CLA9/CLA9T Communication Piece with clothing or hands, as this can create excessive noise.

GUARANTEE

From the moment your Geemarc product is purchased, Geemarc guarantee it for the period of one year. During this time, all repairs or replacements (at our discretion) are free of charge. Should you experience a problem then contact our help line or visit our website at www.geemarc.com. The guarantee does not cover accidents, negligence or breakages to any parts. The product must not be tampered with or taken apart by anyone who is not an authorised Geemarc representative. The Geemarc guarantee in no way limits your legal rights.

IMPORTANT: YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.

DECLARATION: Geemarc Telecom SA hereby declares that this product is in compliance with the essential requirements and other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive 1999/5/EEC and in particular article 3 section 1a, 1b and section 3.

GUARANTEE

Electrical connection: The apparatus is designed to operate from a 230V 50Hz supply only. (Classified as 'hazardous voltage' according to EN60950 standard).

RECYCLING DIRECTIVES

The WEEE (Waste Electrical and Electronic Equipment) has been put in place for the products at the end of their useful life are recycled in the best way. When this product is finished with, please do not put it in your domestic waste bin.

Please use one of the following disposal options:

- Remove the batteries and deposit them in an appropriate WEEE skip. Deposit the product in an appropriate WEEE skip.
- Or, hand the old product to the retailer. If you purchase a new one, they should accept it.

Thus if you respect these instructions you ensure human health and environmental protection.

For product support and help visit our website at www.geemarc.com

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