seca 792 / 795

Deutsch	Seite 3
English	Page 4
Français	Page 5
Español	Página 6
Italiano	Pagina 7

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Precision for Health

With this purchase, you have decided on precision, reliability and a good design.

From the leading manufacturer of medical scales with over 150 years experience.


Today, however, our slogan seca Precision for Health includes much more:

Medical scales, scales for the private use and products to measure condition during sports training.



If you require further information, please do not hesitate to contact us.

And this is how it functions

- Remove the transportation protection: Twist the red protecting pin which can be found underneath the scale, and then pull out.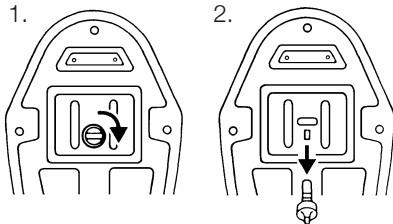

Pull off the protective part above the display, turn it over and put it back in.

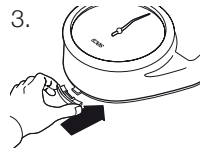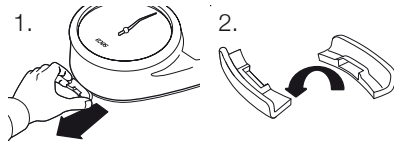3. The protective part now serves as a spacer against a wall or similar.

- Place the scale on a flat floor. Soft carpets can distort the results.
- Set the display to zero. The small wheel can be found directly below the zero.
- Apply your weight evenly to the scale using both feet. Remain still until the weight is displayed.

The scale is so precise that even the pulse beat is transferred to the indicator when you weigh yourself barefoot.

Weighing range: 150 kg. Division: 1000 g.

Weigh yourself correctly

In order that your regular weight control will be meaningful over a longer period of time: Always weigh yourself at the same time during the day – e.g. always first thing in the morning and, of course, without clothes or shoes.

Which weight is the correct one?

As there is no “ideal weight” which can be applied to everyone, it is best to consult your doctor to find out which is the best weight range for you.

Warranty

A three-year warranty from date of delivery applies to defects attributable to poor materials or workmanship. All moveable parts – batteries, cables, mains units, rechargeable batteries etc. – are excluded. Defects which come under warranty will be made good for the customer at no charge on production of the receipt. No further claims can be entertained. The costs of transport in both directions will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be honoured only if the complete original packaging was used for any transport and the scale secured and attached in that packaging just as it was when originally packed. All the packaging should therefore be retained.

A claim under warranty will not be honoured if the equipment is opened by persons not expressly authorised by seca to do so.

We would ask our customers abroad to contact their local sales agent in the event of a warranty matter.

Care

In order to maintain the appearance of the high-quality leather cover, normal leather cleaning agents may be used.

Do not use scouring agents on the chrome or gold surfaces.

The scale ring glass can be cleaned from the inside when removed.

Just turn the ring to the left and then remove. Place back in position and turn to the right.

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