Ⓓ	Inhalator	Gebrauchsanweisung	2-16	Ⓘ	Inalatore	Istruzioni per l'uso	62-76
ⒼⒷ	Inhaler	Instruction for Use	17-31	Ⓓ	Inhalatör	Kullanma Talimatı	77-91
Ⓕ	Inhalateur	Mode d'emploi	32-46	Ⓓ	Ингалятор	Инструкция по применению ..	92-106
Ⓔ	Nebulizador	Instrucciones para el uso	47-61	Ⓓ	Inhalator	Instrukcja obsługi	107-124

Deutsch

Inhalt

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English

Contents

1	About the device	18
2	Signs and symbols.....	18
3	Safety information and warnings ..	19
4	Description of device and accessories	22
5	Initial use	23
6	Operation	24
7	Changing the filter.....	25
8	Disinfecting the nebuliser and accessories	26
9	Cleaning and maintenance	28
10	Disposal	28
11	Troubleshooting	29
12	Technical data.....	30

Included in delivery

For details of the delivery scope, see description of the device and accessories on page 22.

- Inhaler
- Nebuliser
- Compressed air hose
- Sealing cap
- Mouthpiece
- Nose attachment
- Adult mask
- Child mask
- 1 x replacement filter
- Storage bag
- These instructions for use

1 About the device

Dear valued customer

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air.

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

With kind regards

Your Beurer team

Area of application

This nebuliser is a device used for the nebulisation of liquids and liquid medication (aerosols) and for highly effective treatment of the upper and lower respiratory tract.

The nebulisation and inhalation of medication prescribed or recommended by a doctor can prevent against diseases of the respiratory tract, treat the side effects of these disorders and accelerate healing.

For more information on possible applications, consult your doctor or pharmacist.

The device is suitable for inhalation at home. Medicines should only be inhaled when instructed by a doctor.

Inhalation should be performed in a calm and relaxed atmosphere. Inhale slowly and deeply to enable the medicine to reach the small bronchi deep into the lungs. Exhale normally.

2 Signs and symbols

The following symbols appear in these instructions.

-  **Warning** Warning instruction indicating a risk of injury or damage to health.
-  **Important** Safety note indicating possible damage to the unit/accessory.
-  **Note** Note on important information.

The following symbols are used on the packaging and on the nameplate of the device and accessories.

	Application component type B
	Caution! Read the instructions for use.
	Device in protection class 2
	Manufacturer
I	In
O	Out
SN	Serial number

3 Safety information and warnings

Warning

- Before use, ensure that there is no visible damage to the unit or accessories. When in doubt, do not use the unit and contact your dealer or the customer service address provided.
- Using the unit is not a substitute for consultation with or treatment from a physician. Whenever you have any pain or illness, always contact your physician first.
- If you have health concerns of any kind, contact your general practitioner!
- When using the nebuliser, always follow the basic hygiene procedures.
- Always follow your doctor's instructions regarding the type of medicine to be used and the dose, frequency, and duration of inhalation.
- Only use medicines that have been prescribed or recommended by your doctor or pharmacist.
- If the unit does not function correctly or if you begin to feel unwell or feel pain, stop using the unit immediately.
- During use, hold the unit away from the eyes, as some nebulised medicines may cause damage to the eyes.
- Do not operate the unit in the presence of flammable gases.
- This device is not intended to be used by persons (including children) with restricted physical, sensory (e.g. insensitivity to pain) or mental abilities or persons lacking in the required experience or knowledge for safe operation of the device, unless supervised or instructed by a person responsible for their safety.
- Keep children away from packaging materials (risk of suffocation).
- Do not use attachments that are not recommended by the manufacturer.
- Connect the unit only to the mains voltage listed on the type plate.
- Do not dip the unit into water and do not use it in wet rooms. Do not allow any liquid to penetrate the unit.
- Protect the unit from strong impacts.
- Never touch the mains cable or the device with wet hands, this can cause an electric shock.
- Do not pull the mains plug from the socket by the cord.
- Do not pinch or bend the mains cable, pull it over sharp objects, or allow it to hang freely, and protect it against sources of heat.
- If the mains cable or housing is damaged, contact our customer service department or your local dealer.
- Opening the unit carries a risk of electric shock. Disconnection from the mains supply is only guaranteed if the mains plug has been removed from the socket.
- Do not use the unit if it has fallen or been dropped on the floor, been exposed to extreme humidity or is otherwise damaged. In case of doubt, contact our customer service department or your local dealer.
- The IH 25 inhaler must only be used with suitable Beurer nebulisers and with the appropriate Beurer accessories. The use of other nebulisers and accessories can impair therapeutic efficiency and may damage the device.

Safety information and warnings

Important

- Power cuts, sudden faults or other unfavourable conditions may render the unit unusable. You are therefore recommended to keep a spare device or replacement medicine (as agreed with the doctor).
- If adapters or extensions are required, these must comply with the applicable safety regulations. The electrical capacity and the maximum capacity specified on the adapter must not be exceeded.
- The unit and the mains cable must be stored away from sources of heat.
- Do not use the unit in a room in which a spray has previously been used. If this is the case, ventilate the room before commencing treatment.
- Do not allow any objects to enter or obstruct the ventilation openings.
- Do not use the device if you notice an unusual noise.
- For hygiene reasons, you are recommended to limit the use of accessories to individual persons.

- Always disconnect from the mains after use.
- Store the device in a location protected against environmental influences. The device must be stored in the specified ambient conditions.

FUSE

- The unit contains a fuse. This must only be changed by authorised specialist personnel.

Safety information and warnings

General notes

Important

- Use the unit only:
 - On humans
 - For its intended purpose (aerosol inhalation) and in the manner described in these instructions for use.
- **Any improper use can be dangerous!**
- In acute emergencies, first aid takes priority.
- Apart from the medication, use only distilled water or a saline solution in the unit. Other liquids may cause defects to the inhaler or nebuliser.
- This unit is not intended for commercial or clinical use, but only for individual, private household use!

Before using the unit for the first time

Important

- Before you use the unit for the first time, remove all packaging materials.
- Protect the unit against dust, dirt and moisture. Never cover the unit during operation.
- Before use, ensure the device and accessories are at room temperature.
- Place the device on a solid, level surface.
- Do not use the unit in very dusty environments.
- Switch the unit off immediately if it is defective or malfunctioning.
- The manufacturer shall not be held liable for damage or injuries caused by improper or incorrect use.

Repairs


Note

- Never open or attempt to repair the unit yourself, as otherwise proper function is no longer guaranteed. Failure to observe this regulation shall void the warranty.
- If you need to have the unit repaired, contact customer service or an authorised dealer. Send the device and accessories back in clean condition only.



4 Description of device and accessories

Overview of the inhaler




- 1 Carry handle
- 2 Holder for nebuliser
- 3 Mains cable
- 4 On/off switch
- 5 Filter cap with filter
- 6 Hose connection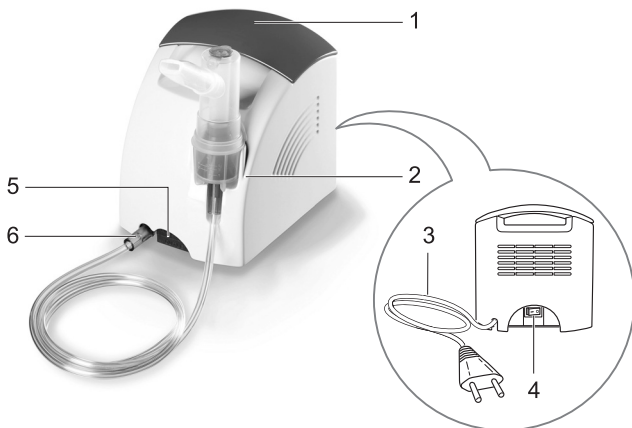

Overview of nebuliser and accessories

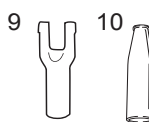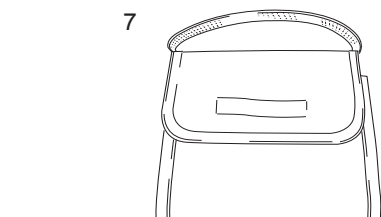- 7 Storage bag
- 8 Adult mask
Child mask
- 9 Nose attachment
- 10 Mouthpiece
- 11 Nebuliser
- 12 Sealing cap
- 13 Nozzle attachment
- 14 Medicine container
- 15 Compressed air hose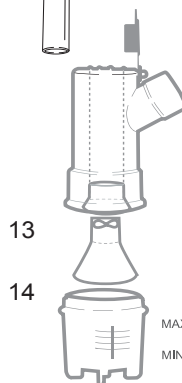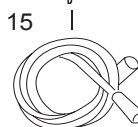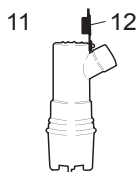

MAX 8 ml
MIN 0,5 ml

5 Initial use

Setup


Remove the unit from the packaging. Place it on a flat surface.

Ensure that the ventilation slits are free.


Before using the unit for the first time

Note:

- Before you use the unit and accessories for the first time, clean it thoroughly. The same applies if you haven't used the device for a prolonged period. See "Cleaning and maintenance" on page 28.
- Attach the hose [15] at the bottom of the nebuliser [11].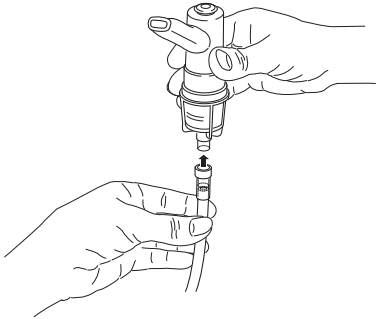

- Connect the other end of the hose [15] to the hose connector [6] on the inhaler by lightly turning it.

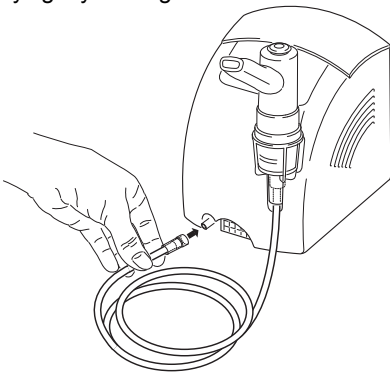Mains connection

Connect the unit only to the mains voltage listed on the type plate.

- Insert the mains cable plug [3] into a suitable socket.
- Push the plug completely into the mains socket.

Note:

- Ensure that there is a socket near to where the unit will stand.
- Lay the mains cable in such a way that no one can trip over it.
- To disconnect the inhaler from the mains after use, first switch off the device and then remove the plug from the socket.

6 Operation

CAUTION:

- Ensure that the nebuliser [11] was cleaned, disinfected and, if applicable, sterilised after its last use (see “Disinfecting the nebuliser and accessories”, page 26).
- Follow the instructions for changing the filter contained in these instructions.
- Before each use, check that the hose connections on the inhaler [6] and the nebuliser [11] are tight.
- Before use, check that the device is functioning correctly. To do this, switch on the inhaler (including the connected nebuliser, but without any medication) for a short time. If air exits from the nebuliser, the device is functioning correctly.

1 Filling the nebuliser

- Pour an isotonic saline solution or the medicine directly into the medicine container [14]. Do not overfill! The maximum fill quantity is 8 ml.
- Only use medicines on the advice of a doctor, and ask for the appropriate inhalation duration and quantity for your needs.
- If the recommended dose of the medicine is less than 2 ml, top up this quantity to at least 4 ml using only isotonic saline solution. Viscous medicines may also require dilution. Always follow the instructions of your doctor.

2 Connecting the accessories to the nebuliser

- Connect the nebuliser [11] to the required accessory part (mouthpiece [10], nose attachment [9] or face mask [8]).

3 Before commencing treatment, pull the nebuliser upwards out of its holder [2].

- Start the inhaler using the on/off switch [4].
- A flow of spray from the nebuliser indicates that the device is functioning properly.

4 Treatment

- Inhale the nebulised medicine deeply.

Note:

During treatment, hold the nebuliser straight (vertical), otherwise the atomisation does not work and correct function cannot be guaranteed.

Important



Essential oils, cough medicines, solutions designed for gargling, and drops for application to the skin or for use in steam baths are wholly unsuitable for inhalation using a nebuliser. These substances are often highly viscous and can impair the correct function of the device and hence affect the effectiveness of its application in the long term.

In the case of oversensitivity of the bronchial system, medicines containing essential oils can sometimes cause acute bronchospasm (a sudden, cramp-like tightening of the bronchi accompanied by breathlessness). Always ask your doctor or pharmacist for advice!

**Note:**

The nebuliser can be used in two different ways:

- With the cap [12] closed, you receive a reduced nebulisation of approx. 0.081 ml/min.
- With the cap [12] open, you receive a standard nebulisation of approx. 0.249 ml/min.

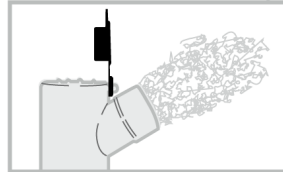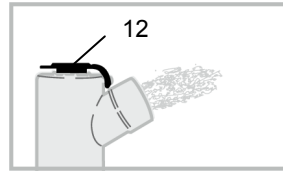**Note:**

The nose attachment [9] must only be used if the cap [12] on the nebuliser [11] is closed.

Note:

The performance specifications may vary for medicines provided in the form of a suspension or which are highly viscous. In these cases, more information can be obtained from the supplier of the medicine.

- 5**
- **End of inhalation:** If the spray is only output sporadically or if the noise changes during inhalation, you can end the treatment.
 - After treatment, turn off the inhaler at the switch [4] and disconnect it from the mains.
 - Once treatment is complete, place the nebuliser [11] back in its holder.

6 Clean the device

See section “Cleaning and maintenance” on page 28 and “Disinfecting the nebuliser and accessories” on page 26.

7 Changing the filter

Check the filter at regular intervals (e.g. after every 10th use). If the filter is very contaminated or blocked, replace the used filter. If the filter has become wet, it must also be replaced with a new one.

**CAUTION:**


- Do not try to clean and re-use the used filter!
- Only use original filters supplied by the manufacturer. Otherwise, your nebuliser may become damaged or effective therapy can no longer be guaranteed.

To replace the filter, proceed as follows:

**CAUTION:**


- First switch off the unit and unplug it.
- Let the unit cool off.

- 1 Pull the filter cap [5] forwards and turn it.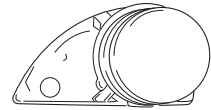

Note:

If the filter remains in the unit when the cap is removed, remove the filter from the unit using tweezers or similar.

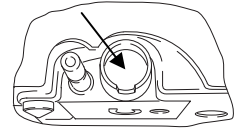- 2 Replace the filter cap [5] containing the new filter.
- 3 Check that the filter cap is firmly in place.

8 Disinfecting the nebuliser and accessories

Warning

Adhere to the following hygiene instructions to avoid possible damage to health.

- The nebuliser and accessories are intended for multiple use. Please note that cleaning requirements differ according to the different application areas:


Notes:

- The nebuliser and any accessories used must be cleaned and disinfected after each use.
- For additional requirements regarding the necessary hygienic preparations (hand washing, handling of medicines or inhalation solutions) in high-risk groups (e.g. cystic fibrosis patients), contact your doctor.
- Ensure that all parts are properly dried following all cleaning and disinfection. Residual moisture or water droplets can result in an increased risk of bacterial growth.
- As with all plastic components, frequent use and cleaning of the accessories can lead to a certain amount of wear. Over time, this can alter the aerosol properties and may eventually

affect the efficiency of therapy. The nebuliser and accessories should therefore be replaced at least once a year.

Preparation

- Directly following each treatment, all parts of the nebuliser and any accessories used must be cleaned of medicine residues and contamination.
 - To do this, dismantle the nebuliser into its component parts.
 - Remove the mouthpiece [10] from the nebuliser.
 - Remove the medicine container [14] from the upper part of the nebuliser.
 - Remove the nozzle attachment [13] from the medicine container [14].
 - Open the sealing cap on the nebuliser.
- The nebuliser is later reassembled in the reverse order.

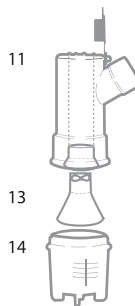Cleaning

If necessary, clean the accessories before use with lukewarm water and dry them using a lint-free cloth. Store the accessories in a clean place.

When cleaning, ensure that all residues are removed and dry all parts carefully. Never use any substances that may be poisonous when in contact with the skin or mucous membranes, or when swallowed or inhaled.

Accessories (with the exception of the hose) can be sterilised using specific products such as the MILTON® method: these must be rinsed until all traces of disinfectant are removed, dried, and stored in a dry and dust-free room. Accessories (with the exception of the hose) can be sterilised by placing them for at least 40 minutes in boiling water (100°C) or at least 20 minutes in an autoclave (120°C). Do not bend or squash the accessories or place any weights on top of them. Do not exceed the specified temperatures, and use a sterile bag. Ensure that the water level is sufficient to prevent nebuliser parts coming into direct contact with the bottom of the pan.

Note:

- In water or steam, the PVC component lose their transparency. This will go away by itself after a few days in a dry atmosphere and does not affect the suitability for use. All accessories can be sterilised up to a maximum of 100 times, after which they must be replaced.

Drying

- Place the individual components on a dry, clean and absorbent surface and allow to dry completely (for at least 4 hours).

Material resistance

- Do not expose the nebuliser and accessories to temperatures above 130°C. Just as all plastic components, a nebuliser is generally a wearing part. The nebuliser can be sterilised up to a maximum of 100 times (see p. 26), after which it must be replaced.
- If the nebuliser is sterilised less than 100 times, it should be replaced at the latest after one year. When choosing a cleaning or disinfecting agent, note the following:
Only use mild cleaning or disinfectant agents in the quantities recommended by the manufacturer.
- When used frequently (more than 3 x daily), we recommend replacing the nebuliser every 6 months to ensure consistently effective nebulisation.

Storage

- Do not store the nebuliser in a damp atmosphere (e.g. in the bathroom) or transport it together with damp objects.
- Store and transport the nebuliser away from direct sunlight.

Storage

- The device and all accessories, e.g. the nebuliser and masks, can be stored together in the storage bag provided.

9 Cleaning and maintenance

Inhaler

Clean the unit occasionally.

Important

Before you clean the unit, always switch it off, unplug it and let it cool off.

Clean using a damp cloth, to which you can apply a little detergent if necessary.

Do not use abrasive detergents and never immerse the unit in water.

Important

- Ensure that no water penetrates inside the unit!
- Do not put the device or accessories in a dishwasher.
- Do not touch the unit with wet hands while it is plugged in; do not allow any water to be sprayed onto the unit. The unit must be operated only when it is completely dry.

- Do not spray any liquids into the ventilation slots. Any liquid that penetrates into the unit can damage the electrical parts or other components of the inhaler and impair the function of the device.

Condensation, care of the hose

Depending on the environmental conditions, condensation may collect in the hose. To prevent against bacterial growth and to ensure effective therapeutic use, this moisture must always be removed. To remove moisture from the hose, proceed as follows:

- Remove the hose [15] from the nebuliser [11].
- Leave the hose plugged into the inhaler [6].
- Run the inhaler until all the moisture has been displaced by the flow of air.
- In the event of heavy contamination, replace the hose.

10 Disposal

In the interest of environmental protection, do not dispose of the device in household waste.

Please dispose of the device in accordance with EU directive **2002/96/EC – WEEE** (Waste Electrical and Electronic Equipment).

If you have any questions, please contact the local authorities responsible for waste disposal.

11 Troubleshooting

Problem/Question	Possible Cause/Remedy
The nebuliser produces little or no aerosol	1. Too much or too little medicine in the nebuliser. Minimum: 0.5 ml, Maximum: 8 ml.
	2. Check that the nozzles are not blocked If necessary, clean the nozzles (e.g. by boiling). Use the nebuliser again. CAUTION: Only poke implements through the fine holes from the underside of the nozzle.
	3. The nebuliser is not held in an upright position
	4. The medicine solution is unsuitable for nebulising (e.g. too thick). The medicine solution should be specified by the doctor.
Output is too low	Kinked hose, blocked filter, sealing cap (see p. 22) is closed, there is too much inhalation solution
Which medicines are suitable for inhalation?	Only the doctor can advise you which medicine to use to treat your condition. Consult your doctor. Essentially all medicines that are suitable and approved for device-based inhalation can be inhaled.
Some inhalation solution remains in the nebuliser.	This is normal and occurs for technical reasons. Stop inhalation if you hear a notable difference in the noise of the nebuliser.
What special steps should be taken for babies and children?	1. In babies, the mask should cover the mouth and nose to guarantee effective inhalation. 2. In children, the mask should also cover the mouth and nose. Nebulisation next to a sleeping person is not suitable because insufficient medicine reaches the lungs. Note: Children should only use the device with help and under supervision of an adult. Never leave a child alone with the nebuliser.
Inhalation with the mask takes longer	This is for technical reasons. Less medicine is inhaled per breath through the mask holes than using the mouthpiece. The aerosol is mixed with ambient air through the holes in the mask.
Why do I need to replace the nebuliser at regular intervals?	There are two reasons for this: 1. To guarantee a therapeutically effective particle spectrum, the nozzle holes should not exceed a certain diameter. Due to the thermal and mechanical stresses, the plastic is subjected to a certain amount of wear. The nozzle attachment [13] is particularly susceptible to this wear. This can also cause changes to the droplet composition of the aerosol, which has a direct effect on the effectiveness of the treatment. 2. You are also recommended to replace the nebuliser on a regular basis for hygiene reasons.
Does each person need their own nebuliser?	This is absolutely necessary for hygiene reasons.
How often must the filter be replaced?	Check the filter regularly for contamination and replace it earlier if necessary.
How often should I disinfect the nebuliser?	Following every use (cleaning and disinfection).


12 Technical data

Dimensions (WHD)	165 x 140 x 155 mm
Weight	1.9 kg
Pressure	approx. 0.8 bar
Nebuliser fill volume	min. 0.5 ml, max. 8 ml
Flow	approx. 4.5 l/min.
Sound pressure	max. 55 dBA (acc. to DIN EN 13544-1:2004-09 section 26)
Mains connection	230 V ~ / 50 Hz / 220 VA
Operating conditions	Temperature: 10 °C – 40 °C Relative humidity: 30 % – 70 %
Storage and transport conditions	Temperature: -40 °C – +70 °C Relative humidity: 10 % – 100 % Atmospheric pressure: 500 hPa – 1,060 hPa

Subject to technical modifications.

Spare parts

Designation	REF
Mouthpiece	162.616
Adult mask Child mask Nose attachment	162.619
Nebuliser Compressed air hose	162.617
Filter	162.618

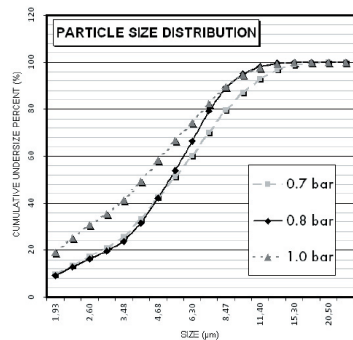These measurements were taken with a physiological solution using the laser-diffraction method. This diagram may not be applicable for suspensions or highly viscous medicines. More information can be obtained from the relevant medicine manufacturer.

Note:

If the unit is used outside of the specifications, proper function is no longer guaranteed! We reserve the right to make technical changes to improve and further develop the product.

This device and its accessories comply with European standards EN60601-1 and EN60601-1-2, as well as EN13544-1, and is subject to special safety measures in terms of electromagnetic tolerance. Note that portable and mobile RF communication equipment can affect this unit. For detailed information, contact customer service at the address provided. The unit conforms to the requirements of the European Directive for Medical Products 93/42/EC, the MPG (German Medical units Act).

ELECTROMAGNETIC COMPATIBILITY

- The device complies with current specifications with regard to electromagnetic compatibility and is suitable for use in all premises, including those designated for private residential purposes. The radio frequency emissions of the device are extremely low and in all probability do not cause any interference with other devices in the proximity.
- It is recommended that you do not place the device on top of or close to other devices. Should you notice any interference with other electrical devices, move the unit or connect it to a different socket.
- Radio equipment may affect the operation of this device.