The image shows a black Welch Allyn Elite Electronic sensor-based stethoscope. The main body is a large, rounded, black plastic piece with three buttons on the side: a power button with a battery icon, a volume up button with a plus sign, and a volume down button with a minus sign. The brand name 'WelchAllyn' is printed vertically on the side. A black cable extends from the top. Below the main body, the two earpieces are shown, which are also black and have a circular design with a white ring. The background is a light blue gradient with abstract white and blue curved lines.

Elite Electronic

Sensor-based Stethoscope
User Manual

WelchAllyn

Elite Electronic Stethoscope

Cardiology Quality for Less

Multi function control

Multi-functional on/off button with light indicator and 3 min. timer function

Cable-port for PC connection and co-listening

Sound enhancement filters clearly marked:
Heart 20-420 Hz
Lungs 350-1900 Hz

Patented high fidelity Sensor for Exceptional Sound Quality

Single 200 hour battery

With new patented ear – piece design rotates for comfort

USER MANUAL

4

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8

MODE D'EMPLOI

12

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16

MANUALE D'USO

20

The Elite Electronic Stethoscope - Innovation in Auscultation Technology.

Thank you for purchasing the Welch Allyn Elite Electronic Stethoscope.

The Elite Electronic features unique patented Sensor Tech® technology that allows you to hear subtle sounds such as diastolic rumbles, gallop sounds, high pitch wheezes/crackles and Korotkof sounds better than conventional stethoscopes. Its independent volume controls allow you to vary the amplification up to a maximum of 93 dB. The Heart & Lung filter settings allow you to focus in on specific frequencies and its special head design helps reduce unwanted noise. Another feature of the Elite Electronic Stethoscope is the sound port located at the rear of the stethoscope (2.5mm stereo) that allows you to connect to an external sound reproduction or recording device for teaching purposes, documenting patient information or for doing simple patient education. Welch Allyn offers several companion products to facilitate these activities such as *The Distributor* and *The Analyzer*. Visit www.welchallyn.com to see all the exciting things you can do with your Elite Electronic Stethoscope!

Unique Features.

- Elite Electronic Stethoscope frequency range: 20-20.000 Hz.
 1. Bell mode : "Heart" 20-420 Hz.
 2. Diaphragm mode : "Lungs" 350-1900 Hz.
- Amplification: Up to 93 dB.
- Weight: 170g.
- Equipped with 2.5mm stereo jack for:
 1. PC connection for sound recordings, patient documentation and analysis of findings (phonocardiogram with ECG)
 2. Interactive teaching of students
 3. Telemedicine
 4. Access to Internet

Sensor Tech® acoustic sensor located in the center of the diaphragm for optimum sound transfer and clarity. Sound is reproduced directly at the earpieces to maximize quality. Earpieces are adjustable.

Chestpiece.

The Elite Electronic Stethoscope chestpiece has a wide base allowing for stable and comfortable auscultation. The actual sensor is located in the center (white area) of the chest piece. This allows you to access body sounds even on small, uneven surfaces (or pediatric patients).

The Meditron sensor is highly directional and can pick up even faint signals very effectively. Varying the pressure applied to the chestpiece during auscultation, is a good way to hear how the Sensor Tech® technology works.

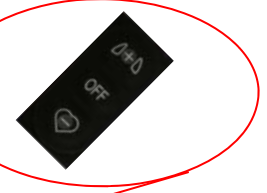
On patients with thick hair in the auscultation area apply some gel on the area first. This will help reduce the potential for picking up the sound hairs make as they move across the sensor.

This will not harm the stethoscope.

Bell/Diaphragm.

Using the buttons on the top of the stethoscope select the desired mode by pressing either the *Heart* symbol (if auscultating the heart) or the *Lung* symbol (when auscultating the lungs).

Note these buttons are multi-function meaning once you select your filter set (*Heart* or *Lung*) the same buttons become your volume controls. Inside the *Lung* symbol is a “+” sign which is used to increase the volume, and within the *Heart* symbol is a “-“ sign used to decrease the volume. An audible “beep” can be heard through the earpieces when the upper or lower volume limits have been reached. To switch from *Heart* to *Lung* mode, you first need to turn the unit off and then select the other mode. Volume will need to be readjusted every time you switch the unit off and back on.

To change to diaphragm mode, (focusing on medium to high frequencies, -350 - 1900 Hz) simply turn the stethoscope off and turn it back on using the upper part of the on-off button.

Note: Diaphragm mode is excellent for higher pitch murmurs, clicks and ejection sounds as well as pulmonary sounds.

Note: Bell mode provides excellent detection for low and medium heart sounds and murmurs such as diastolic rumbles, gallop rhythms, etc.

Volume.

Once the stethoscope is turned on in either *Heart* or *Lung* mode the volume can be adjusted by pressing the “+” sign located within the *Lung* symbol or the “-” sign located within the *Heart* symbol to decrease the volume. The mode you select (*Heart* or *Lung*) will not be affected by adjusting the volume as long as the stethoscope is not turned off. Only by turning it off and then back on can you change the mode. When the volume is at its upper or lower limit a “beep” can be heard through the earpieces. Whenever the unit is initially turned on the volume is automatically set at a midway point. Maximum amplification is 93 dB.

Headset adjustment.

The earpieces can be adjusted by rotating them for individual comfort. The eartips should be pointing slightly forward to align with the ear canals. This will provide an optimum seal for the best sound.

Eartips.

You can clean the ear tips with alcohol or chlorhexidine swabs. The ear tips can be replaced (Part #: 5079-427) by slowly twisting them off. To install the new one's gently twist them on with a clockwise motion.

Caution! Be careful not to close the ear tips with your fingers when twisting. If closed, the ear tips will create air-pressure that may damage the speakers in the earpieces.

Connection to other Meditron products using your Elite Electronic Stethoscope:

5079-430 Meditron Analyzer w/ECG

Allows you to record and listen to patient's heart & lung sounds. The software can display the recording as a phonocardiogram including ECG via a PC. This is ideal for recording, analyzing and documenting your findings. It can be used to document information, share it with colleagues to measure the patient's progress etc.

5079-401 Meditron Distributor

Portable unit designed for co-listening or teaching (up to 5 people). Up to 5 headsets can be plugged into The Distributor allowing each person to hear the same sounds as the person performing the exam. This is an ideal bedside teaching tool and can be a great tool in conjunction with a colleague for a second opinion.

Care and Maintenance.

It is important to clean and disinfect your stethoscope to prevent possible cross-contamination.

1. Chestpiece and earpieces can be cleaned with alcohol, chlorhexidine or with a wet cotton swab. Take care not to let water enter into the stethoscope as this may cause damage.
2. Batteries may be replaced by sliding back the cover underneath the stethoscope head. Expected battery life time is app. 240 hours. (1 x Lithium CR123A)

Warranty.

Your Elite Electronic Stethoscope comes with 1 year warranty.

During the warranty period, Welch Allyn will repair or replace the damaged parts at no charge.

Extended warranty may apply in certain countries and to students. This warranty does not cover damage in shipment or failure caused by tampering, obvious carelessness or abuse. Return the unit for service to any of the Welch Allyn service centers listed below:

- Welch Allyn, European Customer Service,
Navan Business Park, Dublin Road, Navan, County Meath, Ireland
Tel.: 00 353 46 67700. Fax: 00 353 46 67204
- Welch Allyn, Repair Dept., 4341 State Street Road
Skaneateles Falls, NY 13153, USA
Tel.: 1 800 535 6663

For information on international repair sites, go to:

<http://corporate.welchallyn.com/contact/contact.htm>

Ordering Information

5079-405(v)* Elite Electronic Stethoscope

Accessories:

5079-430(v)* Meditron Analyzer Software, (phonocardiogram w/ECG)

5079-401 Meditron Distributor, (portable co-listening device)

5079-414 Meditron Student headsets (to be used with Meditron Distributor)

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- Welch Allyn Italia S.r.l.,