



# **GIMA**

PROFESSIONAL MEDICAL PRODUCTS

---

Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy

[gima@gimaitaly.com](mailto:gima@gimaitaly.com) - [export@gimaitaly.com](mailto:export@gimaitaly.com)

[www.gimaitaly.com](http://www.gimaitaly.com)

## ***CANNULE DI GUEDEL*** ***GUEDEL AIRWAYS***

**Manuale d'uso - User manual**

---



- REF 34431 Guedel airway 40mm
- REF 34432 Guedel airway 50mm
- REF 34433 Guedel airway 60mm
- REF 34434 Guedel airway 70mm
- REF 34435 Guedel airway 80mm
- REF 34436 Guedel airway 90mm
- REF 34437 Guedel airway 100mm
- REF 34438 Guedel airway 110mm
- REF 34383 Guedel airway 120mm
- REF 34439 Mixed Guedel Airways (1pc REF 34431 + 1pc REF 34432 + 1pc REF 34433 + 2pc REF 34434 + 2pc REF 34435 + 1pc REF 34436 + 1pc REF 34437 + 1pc REF 34438)

**Directions for use:**














- 1.Choice the correct size (the correct size for a patient is determined by the distance between the corner of the mouth and the tragus).
- 2.Lubricated and inserted with the curvature initially facing upwards and then rotated halfway through so as to follow the curvature of the tongue.
- 3.In edentulous patients, it may also be inserted right from the outset with its curvature along the curvature of the tongue.
- 4.Once inserted, it is very important to support the jaw to maintain patency of the airway.

**Caution:**

- Read this instruction manual carefully before use.
- This product should be used by trained personnel.


Storage and transport temperature: -20°C to +60°C (-4°F to 140°F).

**Symbols**

	Caution: read instructions (warnings) carefully		Consult instructions for use		Medical Device complies with Directive 93/42/EEC
	Keep in a cool, dry place		Keep away from sunlight		Expiration date
	Manufacturer		Date of manufacture		Disposable device, do not re-use
	Product code		Lot number		Don't use if package is damaged
	Sterilized using ethylene oxide				

M34431-M-Rev.5.01.20

**REF** 34431 - 34432 - 34433 - 34434 - 34435  
34436 - 34437 - 34438 - 34383 - 34439

 Gima S.p.A.  
Via Marconi, 1  
20060 Gessate (MI) - Italy  
Made in Taiwan

**STERILE EO**  0476

