MANUEL UTILISATEUR USER'S MANUAL MANUÁL GEBRAUCHSANWEISUNG MANUALE UTENTE GEBRUIKERSHANDLEIDING KÄYTTÖOPAS KÄSITUSOHJEND LIETOAJA ROOKASGRAMATA دليل الوالد

Tensiomètre & brassard professionnel Professional sphygmomanometer & cuff Tensiómetro y brazalete profesional محاق قفط الدم بطريق الأبراز مهني

CE 0459

Table with multiple columns and rows containing technical specifications, instructions, and contact information in various languages.

Table listing product variants, their designations, and reference numbers in various languages.

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Table showing blood pressure classification with columns for Systolic blood pressure (mmHg), Et/ou, and Diastolic blood pressure (mmHg). It lists Optimal, Normal, and Normal Haute categories with their respective pressure ranges.

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Thank you for purchasing a Mabi tensiometer. When you choose a SPENGLER instrument, you are choosing experience and excellence that has been recognized since 1907.

INSTRUCTIONS FOR USE Please read the leaflet before use and keep it for future reference.

INTENDED MEDICAL USE The Mabi tensiometer is a medical diagnosis professional device (used by health professionals trained in its use) that measures the pressure exercised by blood on arteries by the auscultatory method (a stethoscope is therefore required).

WHAT DOES "BLOOD PRESSURE" MEAN? Blood pressure is the pressure exercised by blood flow on the walls of arteries. The highest pressure, produced in the arteries by each heart beat, is called "systolic pressure" whereas the lowest "diastolic pressure".

WHAT IS "NORMAL" BLOOD PRESSURE? According to the WHO (World Health Organization), normal blood pressure is systolic pressure less than 130mmHg (millimetres of mercury) and diastolic pressure below 90mmHg. However, blood pressure can vary depending on the individual.

Table showing blood pressure classification with columns for Systolic blood pressure (mmHg), And/or, and Diastolic blood pressure (mmHg). It lists Optimum, Normal, Upper normal, Grade 1 HBP, Grade 2 HBP, and Grade 3 HBP categories with their respective pressure ranges.

WARNINGS The warnings in this document highlight conditions or practices which could cause injury, disease or potentially death of a patient. ⚠️ Warning: Possible measurement error. Only use the tensiometer cuffs and Mabi accessories from SPENGLER to avoid measurement errors.

CHOOSE AN CUFF THAT SUITS THE PATIENT'S MORPHOLOGY SPENGLER offers a variety of cuffs for health professionals in using an cuff that suits a patient's morphology, as the size of the cuff impacts the quality of measurements.


Note: The index marker showing the arterial index on the cuff must be within the range shown on Mabi cuffs. When the cuff is positioned on the patient's arm, the "index" marker shown on the cuff must be within the range shown on Mabi cuffs.

Table titled 'Available sizes of cuffs for Mabi tensiometer' with columns for Size of cuff and Arm circumference, listing sizes XS, S, M, L, and XL with corresponding arm circumferences.

SPENGLER Innovation: With its magnetic connectors, the "Life Link®" tube of the Mabi tensiometer means that the cuff can be changed effortlessly in one second.


AMBIDEXTROUS: CHANGE THE CONFIGURATION The Mabi is especially designed to be ambidextrous, change the position of the following parts: - Decompression valve, - Spoon - LifeLink® connection

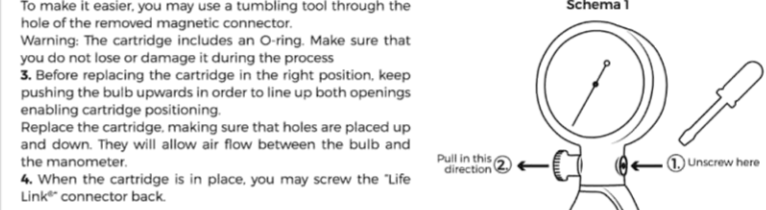Instructions: changing the Life Link® connection 1. Unscrew the bulb and remove it from the device. With the decompression valve closed, unscrew the magnetic "Life Link®" connector with a screwdriver. (Schema 1)

Instructions: changing the bulb 1. Unscrew the bulb and then remove it 2. Unscrew the 2 screws and invert the bulb 3. Tighten the 2 screws 4. Tighten the bulb.

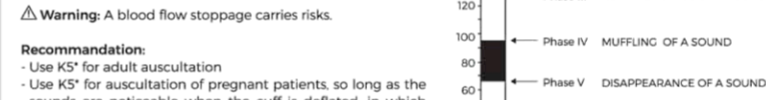USING THE MABI ANEROÏD TENSIO METER - Leave the patient to rest for 5 minutes - Place the patient in a comfortable position, with legs uncrossed, and the back and arm elevated. Ideally without moving or speaking - Check that the marker index for arterial index is within the range shown on the Mabi cuffs

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Environmental specifications: Operating temperature: -10°C to +40°C Operating relative humidity: 20% to 85% Storage temperature: -20°C to +70°C Relative humidity at storage: 20% to 85%

MAINTENANCE AND MAINTENANCE VERIFICATION Comply with current regional regulations for the disposal of manual tensiometers or accessories.

Metrological inspection by an approved body Check accuracy and watertightness - At least once every 2 years (manometer) - At least once per year (cuffs, tube, bulb)

Note: in accordance with international and national metrological standards, the operator or the body responsible for maintenance must ensure traceability of benchmark manometer used for calibration.

Removable parts & accessories: Please contact your approved retailer to purchase the parts and accessories below: - Replacement bulb, - LifeLink® tube, - Cuffs (all sizes).

CLEANING AND DISINFECTION: ⚠️ Warning: Never immerse the manometer in liquid or in a decontamination device. Use a neutral detergent to clean the surface of product. Should disinfection be necessary, use a cloth soaked in 75% alcohol.

⚠️ Warning: Avoid placing your Mabi too close to heat, excessive cold, solvents or greases. Avoid long-term contact with sweat, as the PVC could easily stiffen (bulb and tube). Keep hands dry.

⚠️ Warning: To clean the cuff, do not use alcohol but use warm, soapy water to avoid damaging the printing.

TECHNICAL SPECIFICATIONS: Mabi Manometer Accuracy: +/-3mmHg Displayed range: 0 to 300mmHg Indication type: radial scale Scale grading: increments of 2 mmHg Tube adapter: magnetic adapter for Life Link® connection Pressure generation: bulb Pressure reduction: air release valve Case: ABS Mechanism: copper-beryllium alloy

Life Link® tube PVC tube with magnetic nickel steel connection Mabi cuff Cuff: TPU Tube adapter: magnetic adapter for Life Link® connection.

PRECAUTIONS FOR USE Avoid impacts, particularly on the dial glass. - Do not expose to heat, sun or humidity, dust. - Avoid twisting the tubing while using Life Link® and be careful to hold the connector (not the tube) while unplugging to change cuff.

CONTRAINDICATION The magnetic Life Link® connectors must be held over 5 cm from any ferromagnetic or electronic implant with sensitivity to magnetic fields.

WARRANTY The Mabi tensiometer is guaranteed against any part and labour or operating fault. In accordance with the manufacturer's specifications, under normal maintenance and usage conditions. The 3-year guarantee for the tensiometer (excluding cuff and tube) starts from the date of purchase from an approved retailer.

STANDARD Mabi is compliant with the European reference standard NF EN ISO 81060-1. QUALITY ASSURANCE NF EN ISO 13485

CE MARKING CE certification aneroïd tensiometer provided by CEMED, notified body n. 0459. Class Im device (measurement of blood pressure by auscultatory method) in accordance with European directive 93/42/EEC regarding medical devices.

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