ARGON MEDICAL DEVICES, INC.
1445 Flat Creek Road Athens, Texas 75751 USA
Tel: 800-927-4669;
Tel: +1 903-675-9321
www.argonmedical.com


Emergo Europe, Molenstraat 15, 2513 BH.
The Hague, The Netherlands
Tel: +31 (0)70 345 8570
Fax: +31 (0)70 346 7299

STERILE EO

Eng: BioPince™ Full Core Biopsy Instrument
Spa: Instrumento para biopsias de muestras de centro completo BioPince™
Fre: Trocart de biopsie BioPince™ pour le prélèvement d'échantillons cylindriques
Por (PT): Instrumento de Biópsia para Amostras de Massas Centrais Completas BioPince™
Ger: BioPince™ Vollzylinder-Biopsieinstrument (Full Core)
Dut: BioPince™ Full Core Biopsie-gun
Ita: Strumento BioPince™ per biopsia profonda carotante
Swe: BioPince™ instrument för hel kärnbiopsi
Gre: Εργαλείο Γενικής Κεντρικής Βιοψίας BioPince™
Fin: BioPince™-paksuneulabiopsiainstrumentti
Dan: BioPince™ helkerne-grovnålsbiopsiinstrument
Nor: BioPince™ helkerne-grovnålsbiopsiinstrument
Tur: BioPince™ Tam Core Biyopsi Aleti
Por (BR): Instrumento de biópsia BioPince™ Full Core
Est: BioPince™ täieliku sisebiopsia instrument
Cz: Nástroj na jádrovou biopsii měkkých tkání BioPince™
Hun: BioPince™ teljes magbiopsziás készülék
Lit: BioPince™ stulpelinės plačiašerdė adata biopsijos instrumentas
Pol: Instrument BioPince™ do przeprowadzania biopsji wycinkowej
Sl: Instrument za debeloigelnó biopsijo BioPince™
Sk: Nástroj na úplnú stredovú biopsiu BioPince™
Rom: BioPince™ Full Core Biopsy Instrument
Ar: جهاز لأخذ الخزعات المركزية الكاملة BioPince™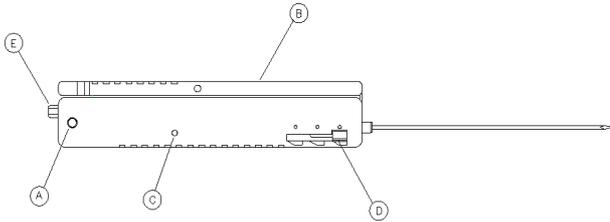

ENGLISH

INDICATIONS FOR USE: The disposable, automatic BioPince™ Full Core Biopsy Instrument is used to obtain multiple core samples from soft tissue such as liver, kidney, abdominal masses, etc.

CONTRAINDICATIONS: Use only for core biopsies of soft tissue as determined by a licensed physician. A physician familiar with the possible side effects, typical findings, limitations, indications and contraindications of core needle biopsy should use the instrument. Physician judgement is required when considering biopsy on patients with bleeding disorders, or receiving anticoagulant medications.

CAUTION: For Single Use Only. Do not attempt to clean or resterilize this product. After use, this product may be a potential biohazard. Handle in a manner, which will prevent accidental puncture. Dispose in accordance with applicable laws and regulations. Verify integrity of needle before cocking instrument and after firing(s). If needle is damaged, appropriately discard the entire device and prepare a new instrument. Do not attempt to open the device.

NOTE: These instructions for the BioPince™ Full Core Biopsy Instrument are NOT meant to define or suggest any medical or surgical technique. The individual physician is responsible for the proper procedure and techniques to be used with this product.

This device was designed, tested and manufactured for single use only. Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury. Do not reuse, reprocess or re-sterilize this device.

INSTRUCTIONS:

1. Prepare the biopsy site appropriately and locate the target area.
2. Advance a BioPince™ Co-axial Introducer Needle, using standard technique, to a position proximal to the lesion. Position the adjustable needle stop proximal to the skin.
Note: If used without co-axial guidance, an incisional dermatotomy is recommended prior to activating the BioPince device through the skin.
3. Remove the stylet from the Co-axial Introducer Needle, leaving the cannula in position.
4. Activate the safety button (A) on the side of the instrument. It will indicate the "SAFE"-position.
5. Cock the instrument by lifting the cocking lever (B) until a "click" is heard. Push the lever down until a second "click" is heard. A YELLOW indicator in the "READY" window (C) indicates that it is cocked.
Note: BioPince™ has three needle stroke lengths-13, 23, and 33mm to provide clinical flexibility. The instrument is shipped in the 33mm position. If an adjustment to stroke length is desired, the instrument must first be cocked. To select the desired stroke length, lift the "ears" on both sides of the stroke length adjuster (D) and move the adjuster to the appropriate position. The 13mm stroke length yields a specimen of 9mm length, the 23mm stroke length yields a specimen of 19mm length, and the 33mm yields a specimen of 29mm length.
6. Advance the BioPince™ stylet to a position proximal to the lesion and release the safety (A). The instrument is ready to "FIRE".
7. Firmly press the firing trigger (E) located at the proximal end of the instrument.
8. Remove the device from the patient. Lifting the cocking lever (B) and pushing it down will automatically expel the specimen. The instrument is now ready for another specimen retrieval.

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