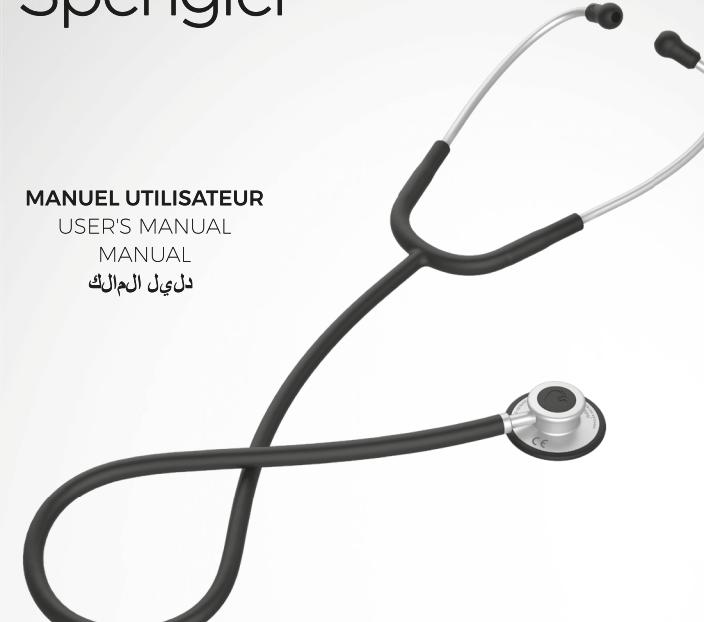
Spengler

MANUEL UTILISATEUR  
USER'S MANUAL  
MANUAL  
دليل المالك

Pulse II

Stéthoscope  
Stethoscope  
Fonendoscopio  
س ساعه طبیب

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FR

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USER'S MANUAL EN

Thank you for choosing the PULSE II stethoscope. Choosing Spengler's Instrument is trusting the experience and recognized excellence since 1907.

PULSE II is the resulting from many years of research and development aiming at providing the best quality, design and comfort to health professionals. This medical device is exclusively manufactured in accordance with the Directive 93/42 EEC, relating to medical devices and is subject to quality controls.

**INSTRUCTIONS FOR USE**

Please read carefully the user manual before using the product and keep it safe for further reference.

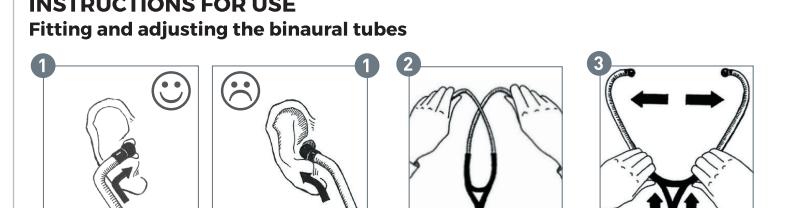
**INTENDED MEDICAL USE**

PULSE II is a professional equipment (can be used by trained health professionals) for medical diagnostic. It may be used for heart auscultation, lungs... as well as measuring the blood pressure.

**Caution:** This device must be used according to the instructions

**INSTRUCTIONS FOR USE**

Fitting and adjusting the binaural tubes

The binaural tubes are designed to automatically adjust to your ears' listening angle. Ensure that the branches of the binaural tubes are pointing forwards, thus matching the direction of the auditory canal [fig. 1].

The binaural tube spring can be adjusted for improved comfort and better adjustment to your morphology.

To increase the spring tension, grasp the branches of the binaural tubes at the level of the earpieces and cross them over [fig. 2].

To reduce the spring tension, grasp the branches at the base of the binaurals and gradually move them apart to complete extension [fig. 3].

**Selecting chestpiece for PULSE II DUAL**

Thanks to the innovative chest-piece indicator, you can see which side of the dual head is activated.

**Using the multi-frequency floating diaphragm**

The PULSE II stethoscope enables you to perform various types of auscultation. The multi-frequency floating diaphragm can be used to listen to both low and high frequency sounds without the need to flip the chestpiece.

**Low frequencies:**

Apply very mild pressure to the chestpiece. The diaphragm is held in place by a PVC ring, enabling low-frequency sounds to resonate.

**High pressures:**

Apply firm pressure on the chestpiece. The diaphragm makes contact with an internal ring, thus allowing high-frequency sounds to be heard.

To alternate between listening to high and low-frequency sounds, simply alter the pressure applied to the chestpiece.

**Using the small bell chestpiece**

The small bell chestpiece allows listening to low frequency sounds.

**MAINTENANCE**

Replacement of earpieces and membranes

**• Earpieces:**

Check regularly that the earpieces are screwed on tight in order to ensure good acoustic quality. To replace the earpieces, remove them and screw on new ones.

**• Diaphragm:**

The diaphragm is held in place by the chill ring. To remove the diaphragm, grip the chill ring between thumb and index finger and pull to remove it from the chestpiece [fig. 4].

Replace the diaphragm on the chestpiece, printed side up, and align the edge of the chill ring with the chestpiece groove. Use your thumbs, attach the chill ring to the chestpiece circumference [fig. 5].

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## UPKEEP AND CLEANING RECOMMENDATIONS

Regular cleaning of your stethoscope and in particular of the earpieces, ensures optimum acoustic performance.

• Do not immerse the stethoscope in liquid.

• Do not sterilize.

• Clean the stethoscope with a cloth soaked in 70% isopropyl alcohol or moderately soapy water.

• Unscrew the earpieces before cleaning them thoroughly by soaking, then drying them. Ensure that no dirt remains inside.

• Keep the stethoscope away from excessive heat or cold, oils and solvents.

• Avoid exposing the stethoscope to direct sunlight.

## GUARANTEE

The PULSE II stethoscope is guaranteed against manufacturing defects (parts and labour) under normal conditions of use, from the date of purchase (invoice date).

• Chestpiece, earpiece, diaphragm and chill rings: 1 year

The guarantee does not apply in the event of use non-compliant with instructions, improper use, modifications made by yourself or by an unauthorised third party, accidental damage and normal wear.

During the guarantee period, any stethoscope parts that present manufacturing defects shall be replaced and/or repaired free of charge, after returning the device to Spengler's after sales service.