
ARGON MEDICAL DEVICES

ARGON MEDICAL DEVICES, INC.
1445 Flat Creek Road Athens, Texas 75751 USA
Tel: +1 (903) 675 9321
Tel: +1 (800) 927 4669
www.argonmedical.com

EC REP

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands
+31 70 345 8570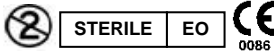

ENGLISH

BioPince™ Ultra Full Core Biopsy Instrument

INDICATIONS FOR USE: The disposable, automatic BioPince Ultra Full Core Biopsy Instrument is used to obtain multiple core samples from soft tissue such as liver, kidney, abdominal masses, etc.

DEVICE DESCRIPTION: The BioPince Ultra Full Core Biopsy Instrument is a disposable, automatic biopsy instrument that cuts a full core sample of soft tissue. The instrument is designed to expel the sample upon re-cocking of the instrument which prepares the instrument for taking another sample (i.e., ready to be fired). The instrument has a safety slide, a ready indicator window and two firing trigger buttons. Additionally, each instrument has an adjustable stroke length (i.e., throw) of three settings:

Instrument Stroke Length Setting	Approximate Sample Length
13mm	9mm
23mm	19mm
33mm	29mm

The instrument is available in 16ga in lengths of 10cm or 15cm; and 18ga in lengths of 10cm, 15cm or 20cm. The instrument can be used with a corresponding co-axial introducer needle.

HOW SUPPLIED:

This product is supplied sterile unless the package has been opened or damaged. Sterilized by Ethylene Oxide (EO). For Single Use Only. Do Not Re-sterilize.

CONTRAINDICATIONS: Physician judgement is required when considering biopsy on patients with bleeding disorders or receiving anticoagulant medications.

WARNINGS:



- Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness and/or injury. DO NOT reuse, reprocess or re-sterilize this instrument.
- This instrument has not been tested for safe use under Magnetic Resonance Imaging systems. The instrument contains ferromagnetic materials.
- The instrument should be used by a physician familiar with the possible side effects, typical findings, limitations, indications and contraindications of core needle biopsy.
- Use only for core biopsies of soft tissue as determined by a licensed physician. Attempting to biopsy hard tissue or masses may reduce the performance, causing damage to the instrument or potentially harm the patient. This instrument is intended for soft tissue biopsy only.
- These instructions for the BioPince Ultra Full Core Biopsy Instrument are NOT meant to define or suggest any medical or surgical technique. The individual physician is responsible for the proper procedure and techniques to be used with this product.

PRECAUTIONS:


- The BioPince Ultra Full Core Biopsy Instrument is available in different needle lengths and gauges. The appropriate length and gauge of the instrument must be selected for the procedure, per physician judgment.
- This instrument was designed, tested, and manufactured for single use only.
- Do not bend the needle.
- Do not contact the needle tips to a hard surface while expelling a sample.
- Verify integrity of needle before cocking instrument and after firing(s). If needle is damaged, appropriately discard the entire instrument (see Disposal section) and prepare a new instrument. Do not attempt to open the instrument.
- Handle in a manner that will prevent accidental puncture.
- The introduction of the needle into the body should be carried out under imaging control (ultrasound, CT, etc.)
- This instrument can be used with an appropriate Argon Co-Axial Introducer Needle. Other manufacturer's co-axial introducer needles have not been tested for compatibility with the BioPince Ultra Full Core Biopsy Instrument.

INSTRUCTIONS:

- Prepare the biopsy site appropriately and locate the target area.
- Using standard technique, advance the co-axial introducer needle, proximal to the lesion. Adjust the needle stop proximal to the skin.
Note: If used without co-axial guidance, an incisional dermatotomy is recommended prior to advancing the BioPince Ultra instrument through the skin.
- Remove the stylet from the co-axial introducer needle, leaving the cannula in position, if co-axial guidance is used.
- Engage the safety slide (A) on the side of the instrument to place in the locked position, if desired.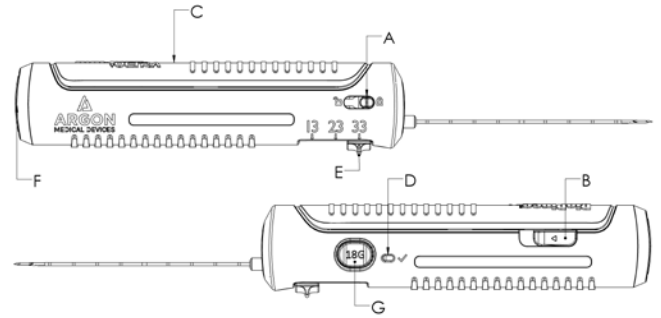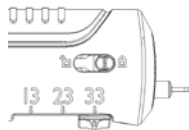

- Cock the instrument by first pushing forward on the release slide (B) until the cocking lever (C) raises, and then lift the cocking lever (C) until a click is heard. Push the cocking lever down until a second click is heard. Green in the ready window (D) indicates that the instrument is cocked.
- The instrument is shipped in the 33mm stroke length position. To adjust the stroke length, push in the stroke length adjuster (E) and slide it until the arrow on the stroke length adjuster aligns with the arrow of the desired stroke length on the instrument.
Note: If an adjustment to the stroke length is desired, the instrument must first be cocked.

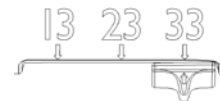- Advance the BioPince Ultra needle into the co-axial introducer needle or percutaneously to a position proximal to the area to be sampled.
- If the instrument is locked, move the safety slide (A) to place in the un-locked position until green is shown in the window. The instrument is ready to fire.
- Firmly press on one of the firing trigger buttons located at the proximal end of the instrument (F) or on the side of the instrument (G).
- Remove the instrument from the patient.
- To expel the sample, push forward on the release slide (B) until the cocking lever (C) raises. Lift the cocking lever (C) until a click is heard, then push the cocking lever down until a second click is heard.
- The instrument is cocked and ready for collecting another sample, if additional biopsies are required. Verify integrity of needle tips before additional biopsies.

DISPOSAL

After use, this product may be a potential biohazard. Dispose in accordance with applicable laws and regulations.

STORAGE

Store at standard ambient temperature.

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BioPince™ is a trademark of Argon Medical Devices, Inc.
© 2017 Argon Medical Devices, Inc. All Rights Reserved.