## WelchAllyn®

**23810 Diagnostic Otscope and 23820 Otscope/Throat Illuminator**

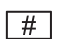
Material No. 720125 Rev. A
© 2013 Welch Allyn, Inc.

## ENGLISH

For guaranteed performance, use only Welch Allyn lamps and specula.

Conforms with: IEC/UL/CSA/EN 60601-1

### Symbols

	Attention. Read Operating Manual for Cautions and Instructions for Use.
	Type BF applied part
	Reorder number
	Product Identifier
	Manufacturer & Date of Manufacture (YYYY-MM-DD)
	Professional use only
	Authorized Representative in the European Community
	Global Trade Item Number
	Consult operating instructions
	Batch code

#### Warnings/Cautions

**WARNING:** Lamps may be hot. Allow the lamps to cool before removing.

**WARNING:** To minimize external otoscope housing temperatures, on-time should not exceed 2 minutes and off-time should not be less than 10 minutes.

**CAUTION:** Halogen lamps are pressurized to provide maximum efficiency and illumination. Mistreatment may cause shattering. Protect lamp surface against abrasions and scratches. Use protective eyeglasses when handling. Verify power is off when replacing lamp. Dispose of lamp with care.

#### Operating Instructions

##### Attaching Otscope Head to power source

Align the cutouts in the Otscope Head base with lugs on power source. Push in slightly and turn 90°. The Otscope is now ready to use. To remove, reverse this procedure. The Otscope Heads fit all 3.5 v. Welch Allyn power sources.

##### Using Focus

(Figure 2)

Use the focus in the Default or Free position.

**Default Focus:** align the green line on the focusing wheel to the green dot on the side of the otoscope.

**Free Focus:** rotate the focus wheel (A) or (B) to zoom in or out.

Zoom in: rotate focus wheel towards longer dashes.

Zoom out: rotate focus wheel towards shorter dashes.

##### Attaching Specula

(Figure 3)

To attach the speculum onto the otoscope, twist the speculum clockwise into the TipGrip (C) until it locks.

##### Removing Specula

To remove the speculum, turn the TipGrip (C) counterclockwise. Or grab the speculum and twist it counterclockwise.

**Note:** Otscope only supports Welch Allyn specula.

##### Using Throat Illuminator for REF 23820

(Figure 4)

- Grasp the otoscope section with the left hand.
- With the right hand, grasp the throat illuminator’s collar at (D) (under the silver ring).
- While holding the throat illuminator collar stationary, twist the otoscope section counterclockwise at (D) until the otoscope head is removed. The throat illuminator is now ready for use.
- To replace otoscope section, reverse the steps.

**Note:** It is important to note that while the throat illuminator section provides an excellent source of general illumination, more than momentary direct skin contact should be avoided due to possible heat build up.

#### Lamp Replacement

(Figure 5)

### Repairs

Repairs should be performed at the factory.

### Cleaning and Disinfection

For cleaning the mirror or lenses, use only a clean, dry optical-grade cloth. Do not use alcohol or other chemical solutions.

The window may be cleaned with a cotton swab or lens paper dipped in alcohol.

The Head housing may be cleaned and disinfected with an appropriate health care low- or intermediate-level cleaner/disinfecting wipe that incorporates either a 1:10 sodium hypochlorite (bleach) solution or isopropyl alcohol as the active disinfection ingredient.

Follow wipe manufacturer’s instructions for appropriate use, contact times and applicable warnings and precautions.

Do not excessively saturate the Head.

Do not immerse Head in any solution.

Do not sterilize the Head. After disinfection, inspect for visible signs of deterioration. If evidence of damage or deterioration is present, discontinue use and contact Welch Allyn or your area representative.

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