Plastic Shoe Horn with Hook

VM952

Usage and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Plastic Shoe Horn from Aidapt. The Shoe Horn is manufactured from the finest materials and components available. When used correctly it is designed to give many years of reliable trouble-free service.

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

INTENDED USE

A conventionally shaped shoehorn with an extended length for easier use. It has a hook for positioning shoes and a hanging loop for storage.

INSTRUCTIONS FOR USE

Using the Shoe Horn requires a certain amount of dexterity:

- Use the hooked end to position the shoe, if necessary.
- Place the front of the foot into the shoe.
- Reverse your grip and hold the hooked end of the Shoe Horn, slipping the wide, rounded end between your heel and the back of the shoe.
- Use the Shoe Horn to lever your heel further into the shoe, whilst it also loosens the back of the shoe to allow for a comfortable fit.
- Carefully pull the Shoe Horn out of the shoe.

CLEANING

Clean your Shoe Horn using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads may damage the product beyond repair and should not be used. Always ensure that you wipe dry the equipment after cleaning.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

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Altai Germany (Altai Europe Ltd.)