FR INSTRUMENTATION ACIER INOXYDABLE CONFORME AUX STANDARDS INTERNATIONAUX

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EN STAINLESS STEEL INSTRUMENTS COMPLYING WITH INTERNATIONAL STANDARDS

Handling recommendations:

- Handle the instruments with care.
- Close the instruments with notch or rack to the first step.
- Handle the instruments following the Best cleaning Practices.
- DO NOT USE WITHOUT PRIOR STERILIZATION.
- The instruments may ONLY be sterilized when undamaged.
- Please choose the correct packaging for sterilization.
- PLEASE CLEAN THE BRAND NEW INSTRUMENTS BEFORE THE FIRST USE.
- Open the instruments for correct cleaning, disinfection and sterilization.
- Please respect the recommendations of the cleaning and disinfecting liquids' manufacturer, in terms of dosage and contact time.
- DRY THE INSTRUMENTS CAREFULLY AFTER CONTACT WITH ANY SUBSTANCE.
- Protocols for pre-disinfection, disinfection and sterilization on www.spengler.fr

Caution:

- Store in a dry and clean place, far from any corrosive substance.
- Handle the sharp instruments with care.
- Keep away from children.

Our stainless steel surgical instruments are guaranteed for 5 years under normal conditions of use by replacing the defective instrument recognized by us.

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Wazirabad Road, Ugoki, Sialkot - Pakistan

Tel: 0092-52-3571214, 3571215, Fax: 0092-52-3571210, 3571211.

Web: www.mdevices.com E-mail: info@mdevices.com

EC Representative OBELIS s.a

Bd.Général Wahis 5310, 0 Brussels, BELGIUM

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Distribué par SPENGLER SAS - 30 rue Jean de Guiramand -
Medical Devices (Pvt) Ltd.