E Barra de apoyo 'Devon'

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Maximum load weight **127 kg** **20 st**

Intended uses

- To compensate for a disability by helping support a person while standing up and sitting down

Instructions for installation

1. These rails should be installed by a person who has suitable qualifications and/or experience in installing this type of product, ensuring the fixtures are appropriate for the specific installation
2. All fixing holes in the wall plate or floor plate (labelled A) must be used. The rail must be fastened securely enough that it is able to support the intended user at point D. The intended user must not exceed the maximum load weight specified right
3. All of the rails can be folded up out of the way when not in use
4. It is recommended that the security of the fixings for the rail should be checked annually or more frequently, depending on usage and location of the rail. Refer back to point 2 for details

Care and Cleaning

The rails can be cleaned with mild detergent and water.

⚠ Cautions

- Ensure that the chosen rail is suitable for use by the intended user
- All rails should be installed by a person who has suitable qualifications and/or experience installing this type of product, who can verify that the rail is fitted correctly
- Before each use, ensure that the rail is secure before putting weight on it.
- On height-adjustable models, ensure the height-adjustment knob is securely tightened
- Ensure that the rail is dry before each use

Product code	Maximum load weight	Height Adjust.	Installation Instructions/Comments
AA2000	67kg	No	Must be permanently attached at the optimum height
AA2002	67kg	No	Must be permanently attached at the optimum height
AA2010	84kg	No	Must be permanently attached at the optimum height
AA2012	84kg	No	Must be permanently attached at the optimum height
AA2018	67kg	Yes	The leg length must be adjusted prior to installation, as follows: (1) Hold the wall plate against the wall and extend the rail to approximately a 45° angle with the wall. (2) Remove the pin clip (C) from the leg, determine the appropriate height, and replace the pin clip. (3) When determining the position of the wall plate for installation, be sure the tip of the leg is securely on the floor.
AA2024	127kg	Yes	Must be mounted to the floor. To adjust rail height, loosen the knob (B) and slide the rail to the desired height. Tighten the knob.
AA2026	127kg	Yes	Must be mounted to the floor. To adjust rail height, loosen the knob (B) and slide the rail to the desired height. Tighten the knob.

These user instructions are available in large print, please contact Patterson Medical Customer Services for a copy. Please retain these instructions for future reference.

Manufactured for Patterson Medical Ltd.

Nunn Brook Road, Huthwaite,
Sutton-in-Ashfield, Nottinghamshire,
NG17 2HU, UK.

UK: Tel: 08444 124 330 Fax: 08448 730 100
www.homecraft-rolyan.com

International: Tel: +44 1623 448 706
Fax: +44 1623 448 784

Patterson Medical Australia

PO Box 1006, North Ryde, NSW Australia 2113.
Tel: 1300 473 422 Fax: 1300 766 473

Patterson Medical France

Zone Industrielle de Montjoy,
11 Avenue de l'Industrie - CS 40712
08013 Charleville-Mézières Cedex, France.
Tel: 03 24 526 115 Fax: 03 24 529 034

Patterson Medical Products, Inc.

1000 Remington Boulevard, Suite 210,
Bolingbrook,
IL 60440-5071, USA.
Tel: 1-800-323-5547 Telefax: 1-800-547-4333

PATTERSON
MEDICAL

Please carefully read the instructions before installation and use of the rail.

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