<b>DE</b> Sprechendes Oberarm-Blutdruckmessgerät Gebrauchsanweisung .....	2
<b>FR</b> Tensiomètre au bras parlant Mode d'emploi .....	17
<b>IT</b> Misuratore di pressione da braccio parlante Instruzioni per l'uso .....	32
<b>NL</b> Bloeddrukmeter voor de bovenarm met gesproken uitvoer Gebruiksaanwijzing .....	47
<b>EN</b> Speaking upper arm blood pressure monitor Instructions for use .....	62

## Inhalt

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## Contents

1. Getting to know your device.....	62
2. Important information.....	63
3. Unit description .....	66
4. Preparing the measurement .....	66
5. Measuring blood pressure.....	68
6. Evaluating results .....	71
7. Saving, displaying and deleting measurements .....	72
8. Error message/trouble shooting .....	73
9. Cleaning and storing the device and cuff.....	73
10. Technical specifications .....	73
11. Mains part.....	74
12. Replacement parts and wearing parts .....	75
13. Warranty/service .....	75

## Included in delivery

- Blood pressure monitor
- Upper arm cuff
- 4 x 1.5 V LR6 AA batteries
- Storage bag
- Instructions for use

## Dear Customer,

thank you for choosing a product from our range. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage, beauty and air. Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Best regards,  
Your Beurer Team

## 1. Getting to know your device

Check that the device packaging has not been tampered with and make sure that all contents are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Services address.

The upper arm blood pressure monitor is used for non-invasive measurement and monitoring of adults' arterial blood pressure.

You can use it to measure your blood pressure quickly and easily, storing the results and displaying the progression of readings together with the average.

You are also warned of possible existing cardiac arrhythmia. The recorded values are classified and evaluated graphically.

## 2. Important information

### Signs and symbols

The following symbols are used in these instructions for use, on the packaging and on the type plate for the device and accessories:

	Caution
	Note Note on important information
	Follow instructions for use
	Type BF applied part
	Direct current
	Disposal in accordance with EC Directive WEEE (Waste Electrical and Electronic Equipment).
	Dispose of packaging in an environmentally friendly manner
	Manufacturer

	Permissible storage and transport temperature and humidity
	Permissible operating temperature and humidity
	Keep dry
	Serial number
	The CE labelling certifies that the product complies with the essential requirements of Directive 93/42/EEC on medical products.
	Certification symbol for products that are exported to the Russian Federation and members of the CIS

### Advice on use

- In order to ensure comparable values, always measure your blood pressure at the same time of day.
- Before every measurement, relax for about five minutes.
- If you want to perform several measurements on the same person, wait five minutes between each measurement.
- Do not take a measurement within 30 minutes after eating, drinking, smoking or exercising.
- Repeat the measurement if you are unsure of the measured value.

- The measurements taken by you are for your information only – they are not a substitute for a medical examination! Discuss the measurements with your doctor, and never base any medical decisions on them (e.g. medicines and their administration)!
  - Using the blood pressure monitor outside your home environment or whilst on the move (e.g. whilst travelling in a car, ambulance or helicopter, or whilst undertaking physical activity such as playing sport) can influence the measurement accuracy and cause incorrect measurements.
  - This device is not intended for use by people (including children) with restricted physical, sensory or mental skills or a lack of experience and/or a lack of knowledge, unless they are supervised by a person who is responsible for their safety or are instructed by such a person in how to use the device. Supervise children around the device to ensure they do not play with it.
  - Do not use the blood pressure monitor on newborns, pregnant women or patients with preeclampsia.  
We recommend consulting a doctor before using the blood pressure monitor during pregnancy.
  - Cardiovascular diseases may lead to incorrect measurements or have a detrimental effect on measurement accuracy. The same also applies to very low blood pressure, diabetes, circulatory disorders and arrhythmias as well as chills or shaking.
  - The blood pressure monitor must not be used in connection with a high-frequency surgical unit.
  - Only use the device on people who have the specified upper arm measurement for the device.
  - Please note that when inflating, the functions of the limb in question may be impaired.
- During the blood pressure measurement, blood circulation must not be stopped for an unnecessarily long time. If the device malfunctions, remove the cuff from the arm.
  - Avoid any mechanical restriction, compression or bending of the cuff line.
  - Do not allow sustained pressure in the cuff or frequent measurements. The resulting restriction of the blood flow may cause injury.
  - Ensure that the cuff is not placed on an arm in which the arteries or veins are undergoing medical treatment, e.g. intravascular access or therapy, or an arteriovenous (AV) shunt.
  - Do not use the cuff on people who have undergone a mastectomy.
  - Do not place the cuff over wounds as this may cause further injury.
  - Place the cuff on your upper arm only. Do not place the cuff on other parts of the body.
  - You can either use the blood pressure monitor with batteries or with a mains part. Please note that data transfer and data storage is only possible when your blood pressure monitor is supplied with power. As soon as the batteries are empty or the mains part is disconnected from the power supply, the blood pressure monitor loses the date and time.
  - To conserve the batteries, the monitor switches off automatically if no buttons are pressed for one minute.
  - The device is only intended for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or careless use.

### Storage and Care

- The blood pressure monitor is made up of precision electronic components. Accuracy of readings and the instrument's service life depend on careful handling.

- You should protect the device from impact, moisture, dirt, major temperature fluctuations and direct exposure to the sun's rays.
- Never drop the device.
- Do not use near strong electromagnetic fields, i.e. keep it away from any radio systems and mobile phones.
- Only ever use the cuffs provided with the monitor or original replacement cuffs. Otherwise erroneous results will be recorded.
- If the instrument is not used for any length of time, we recommend removing the batteries.

### Notes on handling batteries

- If your skin or eyes come into contact with battery fluid, rinse the affected areas with water and seek medical assistance.
-  **Choking hazard!** Small children may swallow and choke on batteries. Store the batteries out of the reach of small children.
- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect batteries from excessive heat.
-  **Risk of explosion!** Never throw batteries into a fire.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a relatively long period, take the batteries out of the battery compartment.
- Use identical or equivalent battery types only.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries.
- Do not disassemble, split or crush the batteries.

### Repair and disposal

- Batteries do not belong in domestic refuse. Used batteries should be disposed of at the collection points provided.
- Never open the instrument. If these instructions are not heeded, the warranty will be null and void.
- Never attempt to repair the instrument or adjust it yourself. We can no longer guarantee perfect functioning if you do.
- Repairs may only be performed by Customer Service or authorized dealers. However, always check the batteries and replace them if necessary prior to making any complaint.
- Please dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).  
If you have any questions, please contact the local authorities responsible for waste disposal.


### Notes on electromagnetic compatibility

- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic

emissions or a decrease in the device's electromagnetic immunity; this can result in faulty operation.

- Failure to comply with the above can impair the performance of the device.


### 3. Unit description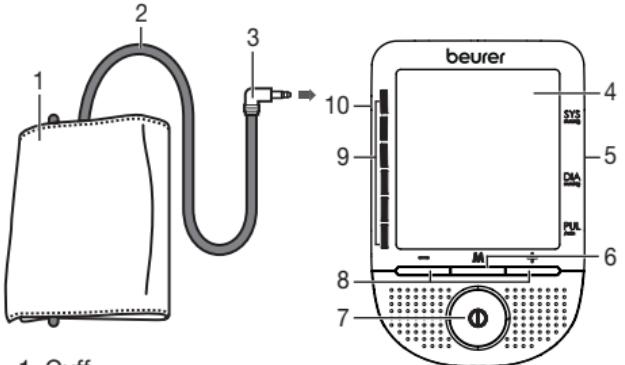

1. Cuff
2. Cuff line
3. Cuff connector
4. Display
5. Connection for mains part
6. Memory button **M**
7. START/STOP button **①**
8. Function buttons **-/+**
9. Risk indicator
10. Connection for cuff connector

### Information on the display:


1. Inflate , release air
2. Systolic pressure
3. Memory display:  
average value (**AVG**),  
morning (**AM**), evening (**PM**)
4. Diastolic pressure
5. User memory
6. Calculated pulse value
7. Cardiac arrhythmia symbol
8. Time and date
9. Risk indicator
10. Memory space number
11. Battery replacement symbol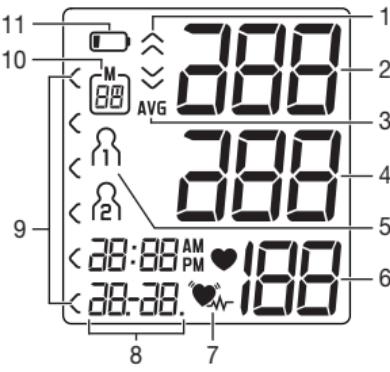

### 4. Preparing the measurement

#### Insert the batteries

- Open the battery compartment lid.
- Insert four 1.5V AA (alkaline type LR6) batteries. Make sure that the batteries are inserted with the correct polarity, according to the label. Do not use rechargeable batteries. The message "The device is ready for use. You may start measuring." is displayed.
- Close the battery compartment lid again carefully.
- With inserted batteries, the unit permanently displays the time and date.

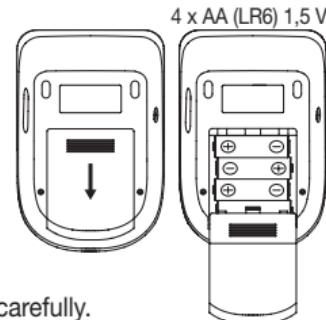If the low battery indicator  and **E5** appear, no further measurements can be performed. The device will announce: "Battery weak." Please replace all batteries. Once the batteries have been removed from the unit, the date, time and language must be set again.

Used batteries should not be disposed of in normal household waste. Dispose of them via your electronics retailer or your local recycling point. You are legally obligated to do so.

## **Battery disposal**

- The empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.
- The codes below are printed on batteries containing harmful substances:

Pb = Battery contains lead,

Cd = Battery contains cadmium,

Hg = Battery contains mercury

## **Set date, time and language**

This menu allows you to set the following functions, one after another.

### **Date → Time → Language → Volume**

It is essential to set the date and time. Otherwise, you will not be able to save your measured values correctly with a date and time and access them again later.

The time is displayed in 24-hour format.

In addition, the device has 4 language settings. The device is set to German when supplied to the customer.

 If you press and hold the function buttons **-/+**, you can set the values more quickly.

**Date/Time**

- Insert the batteries or press and hold the Memory button **M** for 5 seconds. The year flashes on the display.
- Set the year with the function buttons **-/+** and confirm with the Memory button **M**.
- Set the month, day, hour and minutes and confirm each setting with the Memory button **M**.

The language will flash on the display **L 1**.

- You can select the following languages with the function buttons **-/+**:

**L 1** = German

**L 2** = French

**L 3** = Italian

**L 4** = Dutch

**L 0** = Language off

Confirm your selection with the Memory button **M**.

## **Voice output**

*The device is ready for use.  
You may start measuring.*

*Deutsch  
Français  
Italiano  
Nederlands*

**i** **L0** = Language off means that there is no voice output and therefore no acoustic playback.

Volume level **Vo1** flashes on the display.

- You can set the volume of the selected language using the function buttons **-/+**:

**Vo3** = loud

**Vo2** = medium

**Vo1** = quiet

Confirm your selection with the Memory button **M**.

## Voice output


- After using the blood pressure monitor, unplug the mains part from the mains socket first and then disconnect it from the blood pressure monitor. As soon as you unplug the mains part, the blood pressure monitor loses the date and time setting but the saved measurements are retained.

## 5. Measuring blood pressure


Ensure the device is at room temperature before measuring. The measurement can be performed on the left or right wrist.

### Attach the cuff


Place the cuff on to the bare upper arm. The circulation of the arm must not be hindered by tight clothing or similar.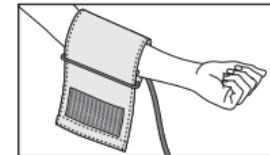

The cuff must be placed on the upper arm so that the bottom edge is positioned 2–3 cm above the elbow and over the artery. The line points to the centre of the palm.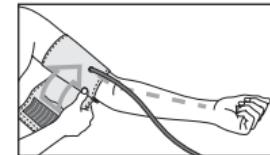

Now tighten the free end of the cuff, but make sure that it is not too tight around the arm and close the Velcro fastener. The cuff should be fastened so that two fingers fit under the cuff.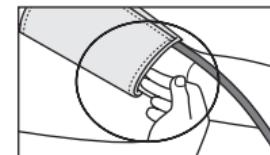


### Operation with the mains part

You can also operate this device with a mains part.


When doing so, there must not be any batteries in the battery compartment. The mains part can be obtained from specialist retailers or from the service address using order number 071.95.

- To prevent possible damage to the device, the blood pressure monitor must only be used with the mains part described here.
- Insert the mains part into the connection provided for this purpose on the right-hand side of the blood pressure monitor. The mains part must only be connected to the mains voltage that is specified on the type plate.
- Then insert the mains plug of the mains part into the mains socket.

Now insert the cuff line into the connection for the cuff connector.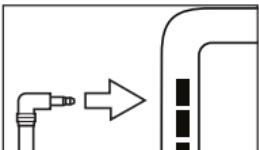

This cuff is suitable for you if the index mark (▼) is within the OK range after fitting the cuff on the upper arm.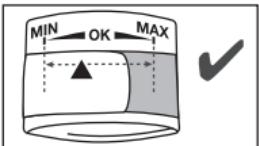

**i** If the measurement is performed on the right upper arm, the line should be located on the inside of your elbow. Ensure that your arm is not pressing on the line.

Blood pressure may vary between the right and left arm, which may mean that the measured blood pressure values are different. Always perform the measurement on the same arm.




If the values between the two arms are significantly different, please consult your doctor to determine which arm should be used for the measurement.

**Caution:** The unit may only be operated with the original cuff. The cuff is suitable for an arm circumference of 22 to 36 cm.

A larger cuff for upper-arm circumferences of 35 to 44 cm can be obtained from specialist retailers or from the service address using order number 163.387.

## Adopt the correct posture

- Rest for approx. 5 minutes before each measurement. Otherwise there may be divergences.
- You can perform the measurement either sitting or lying down. Always make sure that the cuff is on a level with your heart.
- To carry out a blood pressure measurement, make sure you are sitting comfortably with your arms and back leaning on something. Do not cross your legs. Place your feet flat on the ground.
- In order not to distort the result, it is important to keep still during the measurement and not talk.

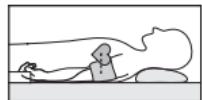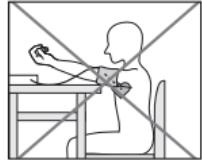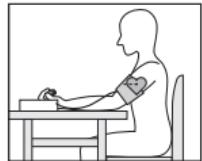## Selecting memory

### User memory

- Switch on the device with the Start/stop **①** button.
- i** When the device is switched off, the time and date are permanently displayed.
- Select the desired user memory by pressing the function buttons **-/+**.

Voice output when voice function is active

The device is ready for use.  
You may start measuring.

User memory 1  
User memory 2

You have 2 memories, each with 60 memory spaces to store the measurements of 2 different people separately.

## Performing the blood pressure measurement

- As described before, attach the cuff and adopt the posture in which you want to perform the measurement.
- Switch on the device with the Start/stop ① button.
- A display check is performed, during which all display segments light up.
- Start the device with the Start/stop ① button. The memory space that will be used is displayed.

### Voice output when voice function is active

### Voice output when voice function is active

*The device is ready for use.  
You may start measuring.*

- The cuff is inflated to 190 mmHg. The cuff's air pressure is slowly released. If a tendency for high blood pressure is already recognised, the cuff is reinflated and the cuff's pressure is increased again. As soon as a pulse is detected, the ❤ symbol flashes.

**i** Measuring can be cancelled at any time by pressing the Start/stop button ①.

- Systolic pressure, diastolic pressure and pulse measurements are displayed.

### Voice output when voice function is active

*Systole ... mmHg  
Diastole ... mmHg*

*Heart rate ... beats per minute*

*According to the WHO guidelines your blood pressure is  
– optimal  
– normal  
– high to normal  
– shows slight hypertension  
– shows medium hypertension  
– shows strong hypertension*

- i** Whilst the message is being displayed, the volume can be adjusted with the function buttons **-/+**.
- **E\_** appears if the measurement could not be performed properly. Observe the chapter on error messages/trouble-shooting in these instructions for use and repeat the measurement.
- The measurement is automatically stored.
- The device switches off automatically after 1 minute.

Wait at least 5 minutes before performing another measurement!

#### Voice output when voice function is active

The symptoms (skipped or premature heart beats, pulse being slow or too fast) can be caused by factors such as heart disease, age, physical make-up, excess stimulants, stress or lack of sleep. Arrhythmia can only be determined through an examination by your doctor. If the symbol  is shown on the display after the measurement has been taken, it should be repeated. Please ensure that you rest for 5 minutes beforehand and do not speak or move during the measurement. If the symbol  appears frequently, please consult your doctor. Self-diagnosis and treatment based on the measurements can be dangerous. Always follow your GP's instructions.

#### Risk indicator:

The measurements can be classified and evaluated in accordance with the following table. However, these standard values serve only as a general guideline, as the individual blood pressure varies in different people and different age groups etc.

It is important to consult your doctor regularly for advice. Your doctor will tell you your individual values for normal blood pressure as well as the value above which your blood pressure is classified as dangerous. The bar chart on the display and the scale on the unit show which category the recorded blood pressure values fall into.

If the values of systole and diastole fall into two different categories (e.g. systole in the High normal category and diastole in the Normal category), the graphical classification on the device always shows the higher category; for the example given this would be High normal.

## 6. Evaluating results

### Cardiac arrhythmia:

This unit can identify potential disruption of the heart rhythm when measuring and if necessary, indicates this after the measurement with the symbol . If the voice function is activated, the device will announce: "A possible cardiac arrhythmia was found." This can be an indicator for arrhythmia. Arrhythmia is an illness in which the heart rhythm is abnormal because of flaws in the bioelectrical system that regulates the heartbeat.

Blood pressure value category	Systole (in mmHg)	Diastole (in mmHg)	Action
Setting 3: severe hypertension	≥ 180	≥ 110	seek medical attention
Setting 2: moderate hypertension	160–179	100–109	seek medical attention
Setting 1: mild hypertension	140–159	90–99	regular monitoring by doctor
High normal	130–139	85–89	regular monitoring by doctor
Normal	120–129	80–84	self-monitoring
Optimal	< 120	< 80	self-monitoring

Source: WHO, 1999 (World Health Organization)

## 7. Saving, displaying and deleting measurements

- Whilst the message is being displayed, the volume can be adjusted with the function buttons **-/+**.

The results of every successful measurement are stored together with the date and time. If there are more than 60 measurements, the oldest measurements are lost.

- Switch on the device with the Start/stop **①** button.

Voice output when voice function is active

*The device is ready for use.  
You may start measuring.*

User memory

Average values

- Select the desired user memory ( $\text{②}$  or  $\text{③}$ ) by pressing the function buttons **-/+**.

- Press the Memory button **M**. The average value of all saved measured values in this user memory is displayed first **AVG**.

- If you press the Memory button **M** again, the average value of the morning measurements for the last 7 days will be displayed (morning: 5 a.m. – 9 a.m., display **AM**).

- If you press the Memory button **M** again, the average value of the evening measurements for the last 7 days will be displayed (evening: 6 p.m. – 8 p.m., display **PM**).

Voice output when voice function is active

User memory 1  
User memory 2

Average count:

Systole ... mmHg

Diastole ... mmHg

Heart rate ... beats per minute

According to the WHO guidelines your blood pressure is ...

- If you continue to press the Memory button **M**, the most recent individual measured values are displayed in turn with the date and time.
- The measured values in the corresponding user memory can be deleted by switching on the device, selecting the desired user memory and pressing the function buttons **-** and **+** simultaneously for 5 seconds.
- The device will go directly into standby mode (clock mode).

## 8. Error message/trouble shooting

In the event of errors, the error message **E\_** appears on the display.

Error messages can occur when

- it was not possible to record the pulse (**E 1**),
- you move or speak during the measurement (**E2**),
- the cuff is fastened too tightly or loosely (**E3**),

Voice output when voice function is active

*Memory space ...*

*Systole ... mmHg*

*Diastole ... mmHg*

*Heart rate ... beats per minute*

*According to the WHO guidelines your blood pressure is ...*

*All the counts in the memory have been erased.*

- errors occur during the measurement (**E4**),
- the pump pressure is higher than 300 mmHg (**E5**),
- the batteries  are almost empty (**E6**).

In such cases, repeat the measurement. Ensure that the cuff line is correctly attached and that you do not move or speak during the measurement.

If necessary, reinsert or replace the batteries.

## 9. Cleaning and storing the device and cuff

- Clean the device and cuff carefully using a slightly damp cloth only.
- Do not use any cleaning agents or solvents.
- Under no circumstances hold the device and cuff under water, as this can cause liquid to enter and damage the device and cuff.
- If you store the device and cuff, do not place heavy objects on the device and cuff. Remove the batteries. The cuff line should not be bent sharply.

## 10. Technical specifications

Model no.	BM 49
Measurement method	Oscillometric, non-invasive blood pressure measurement on the upper arm
Measurement range	Cuff pressure 0–300 mmHg, systolic 50–250 mmHg, diastolic 30–200 mmHg, Pulse 40–180 beats/minute
Display accuracy	Systolic $\pm 3$ mmHg, diastolic $\pm 3$ mmHg, pulse $\pm 5\%$ of the value shown

Measurement inaccuracy	Max. permissible standard deviation according to clinical testing: systolic 8 mmHg / diastolic 8 mmHg
Memory	2x 60 memory spaces
Dimensions	L 139 mm x W 96 mm x H 59 mm
Weight	Approx. 264 g (without batteries)
Cuff size	22 to 36 cm
Permissible operating conditions	+5 °C to +40 °C, 15%–93% relative air humidity (non-condensing)
Permissible storage and transport conditions	-25 °C to +70 °C, ≤93 % relative air humidity, 700–1060 hPa ambient pressure
Power supply	4x 1,5V AA batteries
Battery life	For approx. 250 measurements, depending on the blood pressure level and/or pump pressure
Classification	Internal supply, IPX0, no AP or APG, continuous operation, type BF applied part

The serial number is located on the device or in the battery compartment. Technical information is subject to change without notification to allow for updates.

- This unit is in line with European Standard EN 60601-1-2 (in accordance with CISPR 11, IEC 61000-4-2, IEC 61000-4-3, IEC 61000-4-4, IEC 61000-4-5, IEC 61000-4-6, IEC 61000-4-8, IEC 61000-4-11) and is subject to particular precautions with regard to electromagnetic com-

patibility (EMC). Please note that portable and mobile HF communication systems may interfere with this unit.

- This device is in line with the EU Medical Devices Directive 93/42/EEC, the „Medizinproduktegesetz“ (German Medical Devices Act) and the standards EN 1060-1 (non-invasive sphygmomanometers, Part 1: General requirements), EN 1060-3 (non-invasive sphygmomanometers, Part 3: Supplementary requirements for electro-mechanical blood pressure measuring systems) and IEC 80601-2-30 (Medical electrical equipment – Part 2-30: Particular requirements for the safety and essential performance of automated non-invasive blood pressure monitors).
- The accuracy of this blood pressure monitor has been carefully checked and developed with regard to a long useful life. If using the device for commercial medical purposes, it must be regularly tested for accuracy by appropriate means. Precise instructions for checking accuracy may be requested from the service address.

## 11. Mains part

Model no.	LXCP12-006060BEH
Input	100–240V, 50–60 Hz, 0.5A max
Output	6V DC, 600mA, only in connection with beurer blood pressure monitor.
Supplier	Shenzhen longxc power supply co., ltd
Protection	This device is double insulated and protected against short circuit and overload by a primary thermal fuse. Make sure to take the batteries out of the compartment before using the mains adapter.

	Polarity of the the DC voltage connection
	Double insulated/equipment class 2
Enclosures and Protective Covers	<p>Equipment enclosed to protect against contact with live parts, and with parts which can become live (finger, pin, hook test).</p> <p>The operator shall not contact the patient and the output plug of AC mains adapter simultaneously.</p>

## 12. Replacement parts and wearing parts

Replacement parts and wearing parts are available from the corresponding listed service address under the stated material number.

Designation	Item number and/or order number
Standard cuff (22-36 cm)	163.388
XL cuff (35-44 cm)	163.387
Mains part (EU)	071.95
Mains part (UK)	702.05

## 13. Warranty/service

In case of a claim under the warranty please contact your local dealer or the local representation which is mentioned in the list "service international".

In case of returning the unit please add a copy of your receipt and a short report of the defect.

The following warranty terms shall apply:

1. The warranty period for BEURER products is either 5 years or- if longer- the country specific warranty period from date of Purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Repairs (complete unit or parts of the unit) do not extend the warranty period.
3. The warranty shall not be valid for damages because of
  - a. improper treatment, e.g. nonobservance of the user instructions.
  - b. repairs or tampering by the customer or unauthorised third parties.
  - c. transport from the manufacturer to the consumer or during transport to the service centre.
  - d. The warranty shall not be valid for accessories which are subject to normal wear and tear (cuff, batteries etc.).
4. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

Beurer GmbH • Söflinger Straße 218 • 89077 Ulm, Germany • [www.beurer.com](http://www.beurer.com) • [www.beurer-blutdruck.de](http://www.beurer-blutdruck.de)  
[www.beurer-gesundheitsratgeber.com](http://www.beurer-gesundheitsratgeber.com) • [www.beurer-healthguide.com](http://www.beurer-healthguide.com)

656.28\_BM49-SK1\_2018-10-18\_02\_IM1\_BEU